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

Mphi 5

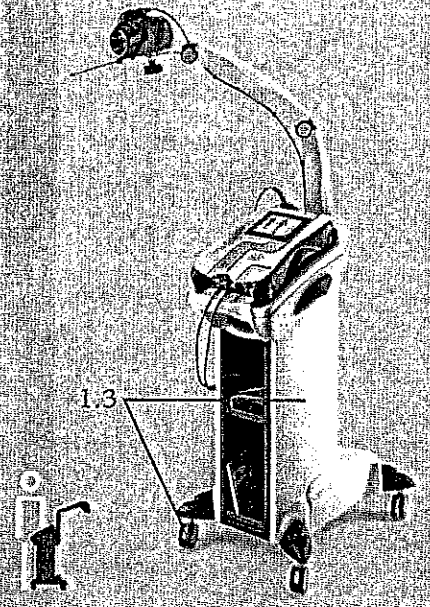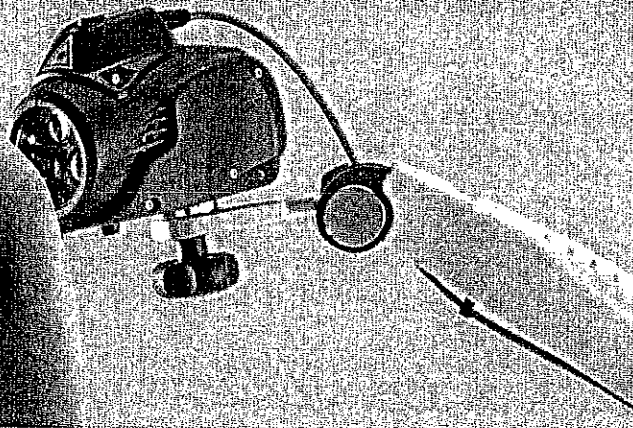Dual treatment
simultaneously


Effective on several body parts.
At the same time.

Mphi 5 allows simultaneous dual treatment. While the operator is using the handpiece on painful areas and trigger points, the Charlie multidiode applicator works uniformly on large areas of tissue, such as major muscle groups or entire joints, in automatic fixed pointer mode.

MLS® Laser Therapy

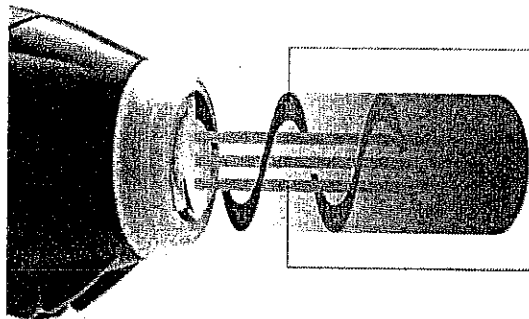
MLS® Laser Therapy comes from ASA's scientific research to overcome the limits of traditional laser therapy and at the same time to exploit the advantages of low and high power through a patented quality impulse.

- Pulsed emission 1.2 (905nm) 1.1
- MLS® Pulse Combined and Synchronized action } 1.1, 1.2
- Continuous / Frequenced emission (808nm) 1.1

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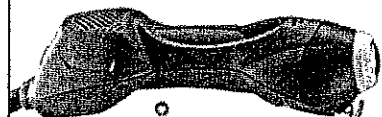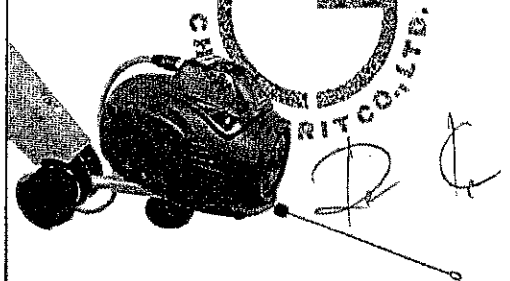
3 MLS® sources - 3 x 25 W Peak Power
Ø5 cm, Target Area 20 cm².

Mphi 5

Technical features

- ▶ Simultaneous management of two independent channels 1.4
- ▶ Multidiode applicator Charlie with optical group composed of 3 MLS® sources power up to 3.3 W - Peak Power 3 x 25 W Target Area Ø 5 cm highlighted by high efficiency red light LEDs 1.4
- ▶ Handpiece with optical group composed of MLS® source power up to 1.1 W - Peak Power 25 W Target Area Ø 2 cm highlighted by high-efficiency red lights LEDs 1.4
- ▶ High resolution LCD color touch screen user interface 1.5
- ▶ Possibility to customize every treatment parameters
- ▶ 4 treatment modalities:
 - A) specific pathologies
 - B) pain management
 - C) anti-edema
 - D) biostimulation
- ▶ MLS® modulation in CPW and FPW mode (Synchronized emission of Continuous Pulsed Waves and Frequenced Pulsed Waves) 1.2
- ▶ Modulation frequency from 1 to 2000 Hz with 1 Hz steps 1.5.2
- ▶ Power level 25%, 50%, 75% and 100% 1.5.4
- ▶ Fixed duty cycle at 50% or variable
- ▶ Treatment duration from 1" to 99'59" with 1" steps 1.5.3
- ▶ Automatic calculation of the emitted energy according to the settled parameters 1.9

ALARMS AND SAFETY FEATURES

- ▶ Laser emission warning light
- ▶ Interlock
- ▶ Acoustic signal selectable by operator
- ▶ Ready to be connected to the remote Laser emission signaling device
- ▶ Date and time
- ▶ Password to operate the device
- ▶ Language option

INCLUDED ACCESSORIES

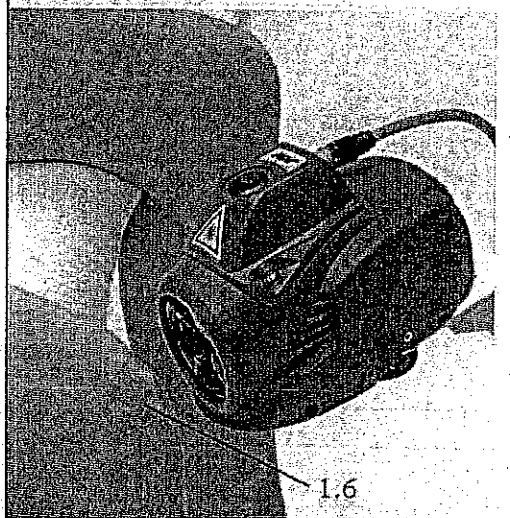
- ▶ N° 2 Laser Safety goggles
- ▶ Unit carrying case
- ▶ Shoulder strap for portable use
- ▶ Table handpiece holder
- ▶ Arm for Charlie
- ▶ Ergonomic trolley designed for holding Mphi, with bumperprotected wheels, a built-in handpiece holder and two storage compartments

SIZE AND WEIGHT

- ▶ 63 x 54 x 85 cm
- ▶ 25 Kg

POWER SUPPLY

- ▶ Lithium polymer battery
- ▶ External power supply 18Vdc

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Acute pain Scar Phase
 Chronic pain Trigger Points

S M L Size


A B Phototype

CPW 700 + 08:408 + 50% 144.096 7.65 1 ON

Hz Time Int Points 1/cm² ASV

Back 1.5.2 1.5.3 1.5.1 1.5.4

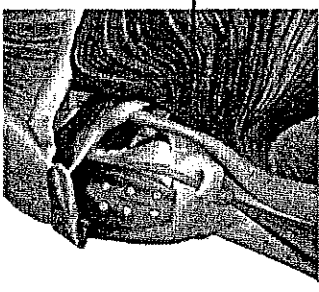1.5.5

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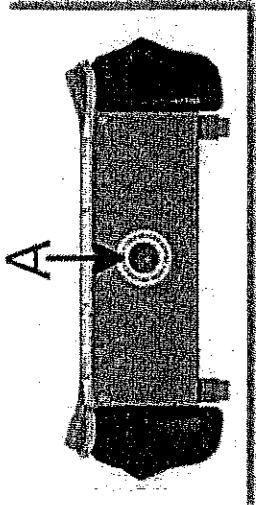
Management software

A - Body type options: S slim, M medium, L large
B - Patient phototype options: A fair-skinned, B dark-skinned
C - Application points, total energy and energy density per point

4.1.3 Emission warning light

The red light emission warning light emits a flashing light signal when Laser emission is standing-by, a constant light when Laser emission is enabled and with the handpieces connected a Laser emission is possible.

Laser emission warning light

- • A - WARNING LASER EMISSION
- • 4.1.4 Warning tone

48 The device is equipped with warning tone to sign the treatment in progress, the beginning, the interruption or the end of treatment.

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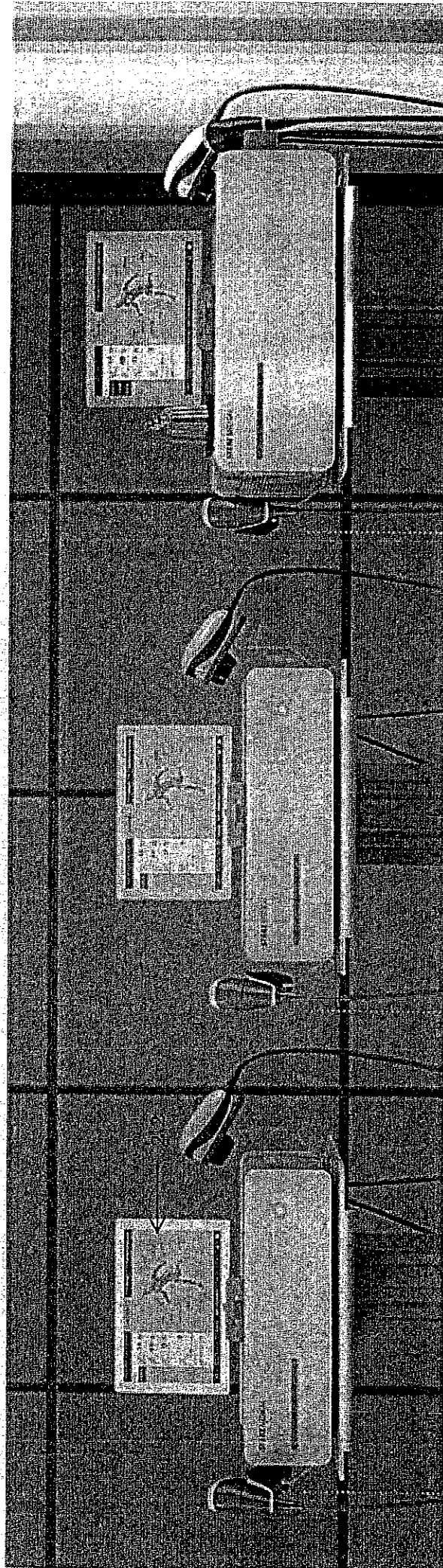
MASTERPULS® MP50 »ultra«

Sophisticated basic module

- General data**
- Dimensions without touch screen (W x H x D): 426 x 144 x 340 mm
 - Weight: 9.5 kg
 - Compressor: integrated
 - 10" touch screen (optional) 2.2.1

- Radial pressure wave**
- Pressure: max. 4 bar_g, frequency: 1 – 17 Hz (depending on pressure)
 - V-ACTOR® »HF« handpiece (optional): 1 – 21 Hz

MASTERPULS® MP100 »ultra«

High-energy compact module

- General data**
- Dimensions without touch screen (W x H x D): 426 x 144 x 340 mm
 - Weight: 10.5 kg
 - Compressor: integrated
 - 10" touch screen (optional)

- Radial pressure wave**
- Pressure: max. 5 bar_g, frequency: 1 – 21 Hz (depending on pressure)
 - »Skin Touch« function
 - V-ACTOR® »HF« handpiece (optional): 1 – 50 Hz

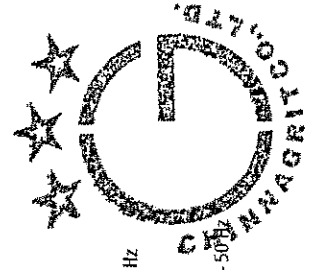
MASTERPULS® MP200 »ultra«

Triple technology »Deluxe« module

- General data**
- Dimensions without touch screen (W x H x D): 466 x 187 x 454 mm
 - Weight: 23 kg
 - Compressor: integrated
 - 10" touch screen (optional)

- Radial pressure wave**
- Pressure: max. 5 bar_g, frequency: 1 – 21 Hz
 - »Skin Touch« function
 - VACU-ACTOR® technology
 - V-ACTOR® »HF« handpiece (optional): 1 – 50 Hz

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Touch Screen Panel

The ideal add-on to the MASTERPULS® »ultra«

The optional 10" touch screen panel is connected with a USB cable to the pressure wave device and adds on valuable features: Apart from the additional device control it offers a patient management system as well as treatment parameters recommended by experienced users and supported by videos and images. These parameters can be selected and adapted. The integrated Visible Body software allows the user to

immerge deeply into the muscular structures and additionally also in the macroscopic and microscopic levels of the human body. Detailed definitions and information about the human anatomy support the user. Freely moveable and rotatable 3D models allows the visualization of muscle movements, pathologies and diseases. Thus, a novel interaction between practitioner and patient will be enable.

Highlights

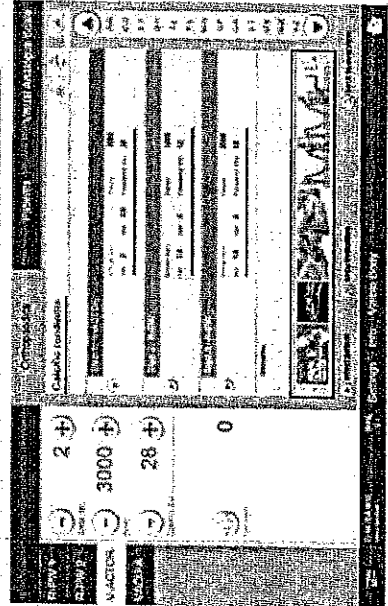
- 10" touch screen panel (optional)
- Patient management system with treatment history
- Treatment parameters supported by videos and images
- New Visible Body® – Digital Human Anatomy Atlas: macroscopic and microscopic 3D models of the human anatomy

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
2.2.3


Touch screen – treatment parameters with application videos


Touch screen – overview of pain zones

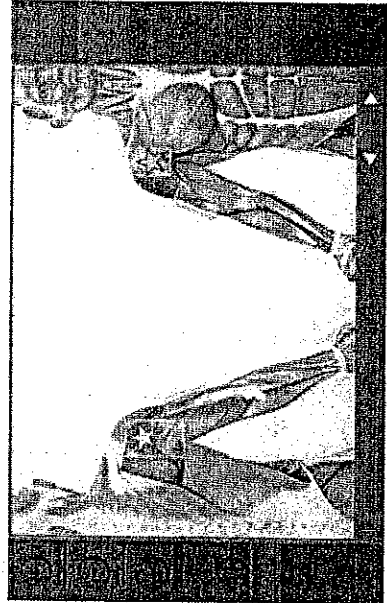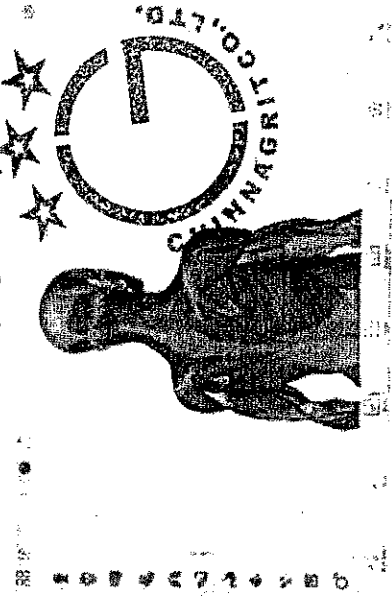Touch screen – new Visible Body® digital anatomy atlas

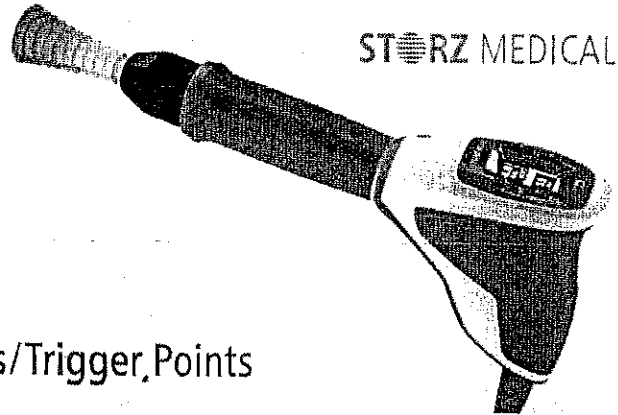Transmitters for R-SW Handpiece »FALCON«

STORZ MEDICAL

2.1.7 Radial Transmitters – Enthesopathies/Trigger Points

A6 »AkuSt«

- Ø 6 mm
- Shock wave acupuncture
- Penetration depth: 0 – 20 mm
- Intensity: ●●●●●●●●●●

F15 »White Soft«

- Ø 15 mm
- Superficial pain regions, muscles of mastication, cervical spine
- Penetration depth: 0 – 30 mm
- Intensity: ●●○○○○○○○○

T10 »Finger«

- Ø 10 mm
- Trigger points, deep-lying enthesopathy
- Penetration depth: 0 – 40 mm
- Intensity: ●●●●●●●●●●

R15 2.1.7 »Basic Energy«

- Ø 15 mm 2.1.7
- Any type of enthesopathy
- Penetration depth: 0 – 35 mm
- Intensity: ●●●●●●●○○○

DI15 2.1.7 »Deep Impact«

- Ø 15 mm 2.1.7
- Deep-lying pathologies, chronic conditions, high-energy application
- Penetration depth: 0 – 50 mm
- Intensity: ●●●●●●●○○○

C15 »CERAmA-x«

- Ø 15 mm
- Any type of enthesopathy
- Penetration depth: 0 – 30 mm
- Intensity: ●●●●●●●●●●

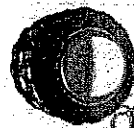
D-ACTOR® Transmitters – Myofascial Treatment

D20-S 2.1.7 »Basic Oscillator«

- Ø 20 mm 2.1.7
- Muscle tissue and connective tissue
- Penetration depth: 0 – 50 mm
- Intensity: ●●●●○○○○○○

D20-T »Golden Oscillator«

- Ø 20 mm
- For professional athletes, high-energy application
- Penetration depth: 0 – 50 mm
- Intensity: ●●●●○○○○○○

D35-S »Large Oscillator«

- Ø 35 mm
- Muscle tissue and connective tissue, large muscles
- Penetration depth: 0 – 60 mm
- Intensity: ●○○○○○○○○○○

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Special Transmitters – Enthesopathies/Trigger Points

Ro40 »Beam«

- Ø 15 mm
- Any type of enthesopathy
- Penetration depth: 0 – 35 mm
- Intensity: ●●●●●●●○○○

B15 »White Tip«

- Ø 15 mm
- Any type of enthesopathy
- Penetration depth: 0 – 30 mm
- Intensity: ●●●●○○○○○○

ATLAS »Soft Tip« (after Di Maio/Swart)

- Ø 15 mm
- Any type of enthesopathy
- Penetration depth: 0 – 10 mm
- Intensity: ●○○○○○○○○○○

Abbreviations

The following abbreviations are used in this manual:

R-SW	R-SW Handpiece / Radial Shock waves 2.1
ESWT	Extracorporeal Shockwave Therapy
TrST	Trigger Point Shockwave Therapy
VAS	Visual Analogue Scale
PA	Potential equalisation
LCD	Liquid Crystal Display
HP	Handpiece

Warning notes

This manual contains warnings, safety instructions and specific operating instructions in accordance with liability regulations.

DANGER refers to a situation of acute danger which, if not avoided, could lead to serious or fatal injury.

⚠ DANGER

The source of the danger is stated here.

These are the possible consequences!

- ▶ The instructions for avoiding the danger are given here.

WARNING refers to a situation of potential danger which, if not avoided, could lead to serious injury.

⚠ WARNING

The source of the danger is stated here.

These are possible consequences!

- ▶ The instructions for avoiding the danger are given here.

CAUTION indicates that incorrect operation could lead to minor injuries.

⚠ CAUTION

The source of the danger is stated here.

These are the possible consequences!

- ▶ The instructions for avoiding the danger are given here.

NOTICE indicates that incorrect operation could lead to damage to the device.

NOTICE

The source of the danger is stated here.

These are possible consequences!

- ▶ The instructions for avoiding the danger are given here.

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2.1 Physical principles

2.1.1

The MASTERPULS® MP50 is a ballistic pulse generator operated by compressed air. The pulses in the MASTERPULS® MP50 are generated with a precision ballistic mechanism in the handpiece. A projectile is accelerated by compressed air. The motion and weight of the projectile produces kinetic energy. When the projectile impacts against an immovable surface, the pulse transmitter, this kinetic energy is converted into sound energy. This acoustic pulse is transmitted with the help of a gel directly into the tissue to be treated.

Physically speaking, these are radial pressure waves. The applied pressure pulse propagates radially within the tissue and has a therapeutic effect, particularly on areas of the tissue that are near the surface.

Medical devices operating on the basis of the above principle are generally referred to as radial shock wave systems in modern medical literature.

2.1.1 Indications

- Calcaneal spur/plantar fasciitis
- Shoulder pain with or without calcifications
- Achillodynia
- Trochanteric bursitis/proximal iliotibial band friction syndrome
- Radial/ulnar humeral epicondylitis
- Patellar tip syndrome
- Tibial edge syndrome
- insertion tendonitis in general
- Treatment of deep-lying muscle trigger points
- Treatment of superficial muscle trigger points, myofascial trigger points
- Insertion tendonitis near the surface (paratendinary area)
- Chronic back pain (cervical/lumbar parts of vertebral column)
- Spastic muscle paralysis / spasticity / increased muscle tone

2.1.2 Contraindications

Treatments with the MASTERPULS® MP50 are not permitted in the following cases:

- Brain or spinal column in the treatment area
- Pregnancy
- Malignant tumour in the treatment area

WARNING

Treatment outside the intended areas

may result in increased side effects or injury.

- ▶ The MASTERPULS® MP50 must not be used to treat regions near large nerves, vascular tissue or the head (except in the facial area).

4.4.2 Parameter selection and counter display 2.2.5

The PARAMETER SELECTION field is used for displaying and setting the treatment parameters.

Operating mode R-SW

This is where you define the energy level as well as the number and frequency of the pulses before each treatment.

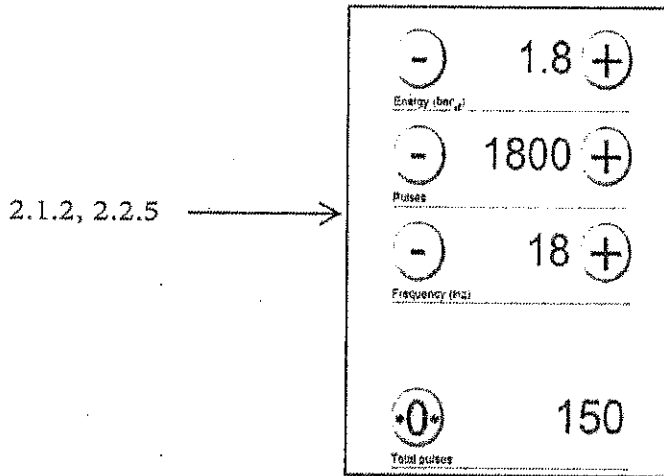
Fig. 4-6 PARAMETER SELECTION area - Operating mode R-SW

Symbols	Meaning
	Set energy level: \oplus increase / \ominus reduce The set energy level is displayed. 2.1.2, 2.2.5
	Set nominal pulse value: \oplus increase / \ominus reduce The set number of pulses is displayed. 2.2.5 If the nominal pulse value set to '-' this means pulse limitation is switched off.
	Set the nominal frequency: \oplus increase / \ominus reduce The set frequency is displayed. 2.1.3, 2.2.5 Setpoint frequency set to '-' means pulses are emitted one at a time.
	Display of the number of pulses emitted. The \odot reset button is used to set the display to 0.

Tab. 4-2 Setting parameters

The desired energy level, nominal pulse value and frequency can be reached more quickly by holding down the \oplus / \ominus button.

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4.2 Operating options

The following options are available for operating the MASTERPULS® MP50:

- via the handpiece display
- via the tablet screen.

4.3 Operation via the handpiece

The MASTERPULS® MP50 can be controlled directly using the handpiece. Corresponding setting buttons can be used for selecting the treatment parameters. The display shows which setting has been selected.


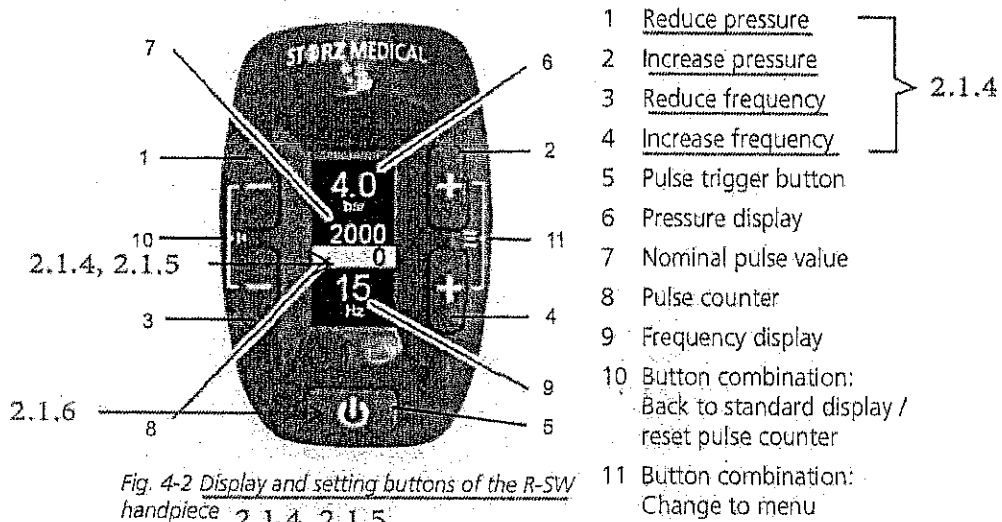
Fig. 4-2 Display and setting buttons of the R-SW handpiece 2.1.4, 2.1.5

The complete overview of functions and description of the handpiece can be found in the chapter on the R-SW handpiece.

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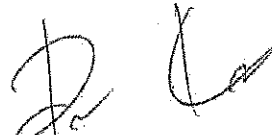To display only those indications for a particular treatment zone

- Press the marker for the treatment zone.

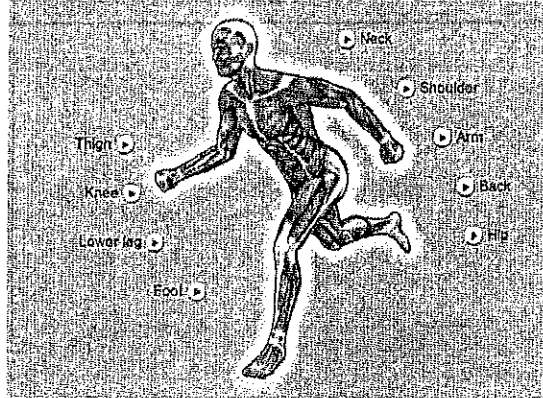
Fig. 4-15 Selectable treatment zones

⇒ The alphabetically sorted list of indications for this treatment zone is opened.

Loading indications

- Load an indication using the button in front of it.

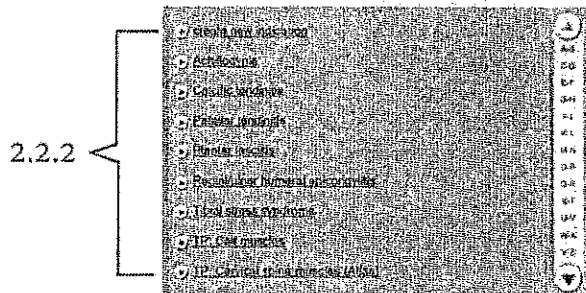
Fig. 4-16 List of stored indications

The following information is displayed about this indication in the overview:

- Treatment steps
- Treatment notes (remarks)
- Treatment pictures

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4.5.11 **Patient treatment report 2.2.4**

Each treatment of a patient can be recorded in a treatment report and stored.

4.5.11.1 **Creating new patient data**

- Press **PATIENTS** in the top menu bar.
⇒ The alphabetically sorted list of patient data is opened.

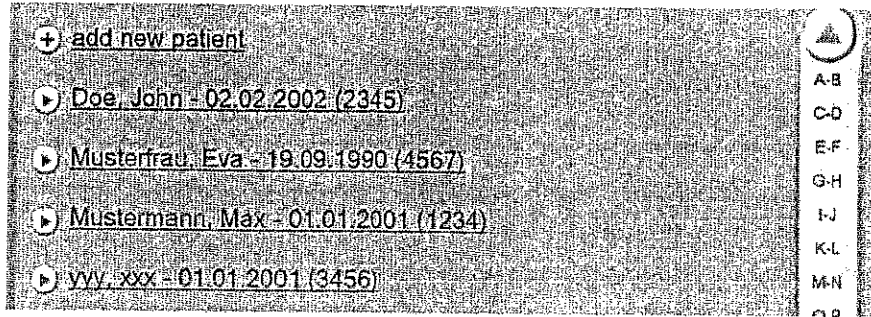
Fig. 4-24 List of stored patient data

- Press **+ NEW PATIENT**.
⇒ A window with a keypad and text boxes for the patient data is opened.

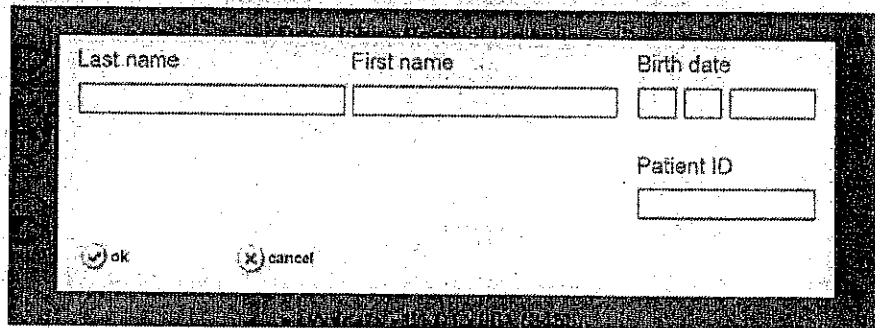
Fig. 4-25 Creating a new patient

- Enter the data.
- Save the entry by pressing **OK**.
- You can edit your new patient dataset to
 - Store notes
 - Attach treatment pictures

For this, please read Chapter 4.5.10.1 **STORE TREATMENT NOTES** and Chapter 4.5.10.2 **LOADING PICTURES AND/OR VIDEOS.**

- Press the **CHANGE PATIENT** button to set the field you want to change to Edit mode.
- Carry out your changes and save the entry by pressing **OK**.

Handwritten signature

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4.5.11.2 Loading patient data

- Press **PATIENTS** in the top menu bar.
⇒ The alphabetically sorted list of patient data is opened.

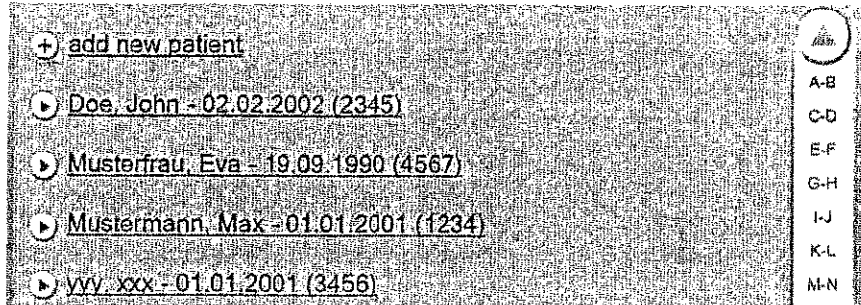
Fig. 4-26 List of stored patient data

Navigating in the list

Use the navigation bar on the right edge of the screen to move within the list.

- Press the or button to scroll up or down,
or
- Call up a list filtered by initial letters by selecting the corresponding letter pairs.
- Load a patient using the button in front of it.

⇒ The following information is displayed about this patient in the overview:

- Name, date of birth and patient number
- Notes
- Pictures
- Treatments performed

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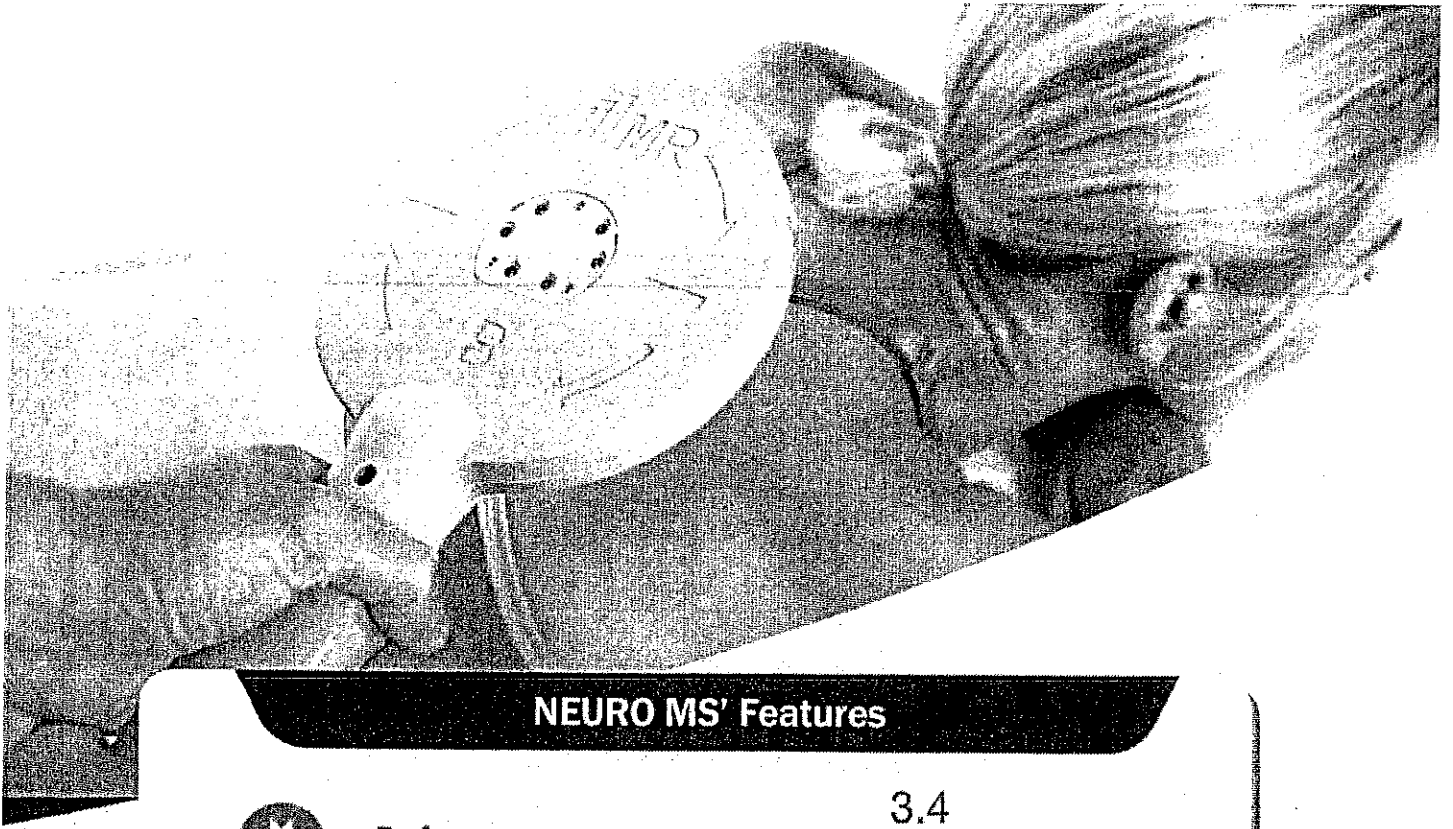
NEURO MS' Features

3.4

Performance NEURO MS features the world's strongest output an electromagnetic stimulation device can offer. Experiences its smooth yet powerful stimulation.

Stability NEURO MS is used in various environment with different temperatures, humidity levels and electrical stability. You can use NEURO MS stably anywhere you are. We've developed and introduced the coil internal circulation water cooling system into NEURO MS and maximized the coil cooling efficiency. Its water cooling system allows the coils to maintain a constant temperature and the device to operate with maximum efficiency.

Economical NEURO MS has an impeccable design and structure and, a very low failure rate. Also, costs for the parts can be saved because NEURO MS does not have any expandable components.

Convenience Patients do not have to undress for their treatment and physicians can easily operate the device without a complicated training process.

Technology We currently own a patent for a pulse-varying power supply system which is a core technology required for an electromagnetic stimulation system and a cooling systems in Korea. (Patent Pending in the U.S.)

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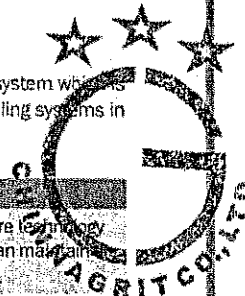
Pulse-Varying Power Supply System

We currently own a patent for a pulse-varying power supply system which is a core technology required for an electromagnetic stimulation system and by using this system, we can maintain a longer pulse rise time and create various forms of pulses.

Cooling System for Electromagnetic Stimulator & Cooling System with a Thermoelectric Module

We currently have the technology and the patent for a cooling system for the coil that generates magnetic fields. By using this technology and systems, we were able to solve the heating problem in the original MS System and build an electromagnetic stimulator that can be operated stably for a long time.

coil probe stimulating electromagnetic field, arm to hold coil probe and power cable etc.

5. 2. 2. Fail-safe

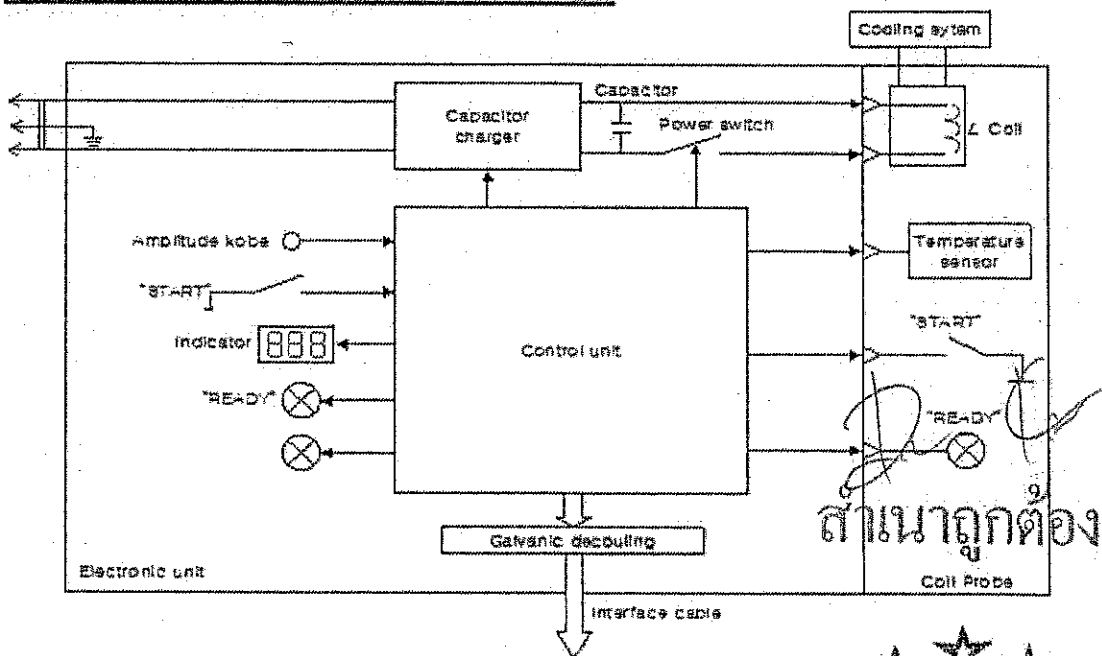
Power should be shut down by fracture of fuse (250V, 10A) when overcurrent flows in the equipment through the power line.

Head of coil probe has a temperature indicating sensor, so when it's temperature is too high or cooling chiller doesn't work, the equipment will be stop immediately. In the case of overheated temperature, coil temperature indicator's lights on the red segment.


5. 2. 3. Fundamentals

3.1

NEURO MS is intended for generation of electromagnetic pulse of high intensity and it includes the electronic unit and the coil

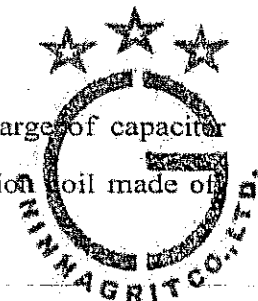Magnetic stimulator principle of operation is based on discharge of capacitor system of high voltage and high current intensity on stimulation coil made of

1. Principles

3.2

NEURO MS is intended for therapeutic impacts by electromagnetic pulses to muscle, bone and peripheral nervous system.

NEURO MS uses the electromagnetic pulses of short duration which can easily penetrate through clothes, bones and soft tissues. NEURO MS can be applied in neurology, traumatology and orthopedic and areas of medicine for:

- Treatment of peripheral nervous system affections;
- Treatment of vascular and vertebra genic pathologies, neuropathies, demyelinating diseases;
- Treatment of traumatic injuries of bones, spine, muscles, joints, spine and its complications;
- Painless stimulation of peripheral nerves;

NEURO MS can be used in patient care institutions, neurosurgical clinics, experimental laboratories of scientific and research institutes.

It is a medical decision, whether the risk of use clearly outweighs any possible benefit in the individual case.

General contra-indications for NEURO MS application are acute arterial hypotension, complex blood diseases, susceptibility to bleedings, thrombophlebitis, thromboembolism, bone fractures before immobilization, diffuse toxic goiter of the III stage, acute suppurative inflammation, abscesses and phlegmon before dissection and draining of cavities, gall-stone disease, epilepsy, pregnancy. Do not apply the stimulus to a pregnant woman's head, neck, or abdomen.

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
ATTENTION!

Electromagnetic field emitted by the stimulators can damage the implanted cardiac pacemakers and cochlear implants.


4. Specifications

1 Magnetic stimulation		
Magnetic induction amplitude(Intensity)	0~3 Tesla	3.3
Shape of stimulation pulse	Biphase	3.4.3
Pulse duration	250 μ s	3.4.4
2 NEURO MS Operation		
Stimulation intensity	0~100 %	
Stimulation frequency	0.1~30 Hz	3.4.2
Stimulation train duration	0.5~30 s	3.4.5
Pauses between trains	0~30 s	3.4.6
Session time	0.5~30 min	3.4.7
3 Stimulation mode 3.4.8		
Single mode	1	
Continues mode	1	
Auto mode	5	
Memory mode	5	
4 Voltage, Power		
Supply voltage	220V, 50/60 Hz	
Power consumption, not more than	Max. 1.2KVA	3.4.9
5 Dimension		
Main body(W*L*H, mm)	365*465*1035	ค่ามาตรฐาน
Total weight	56.5 kg	
Safety type	B	

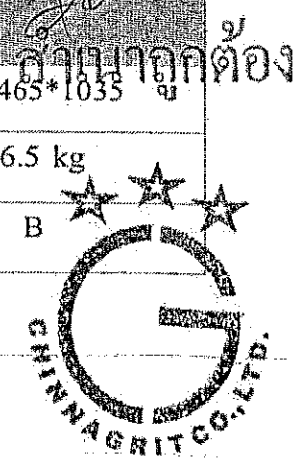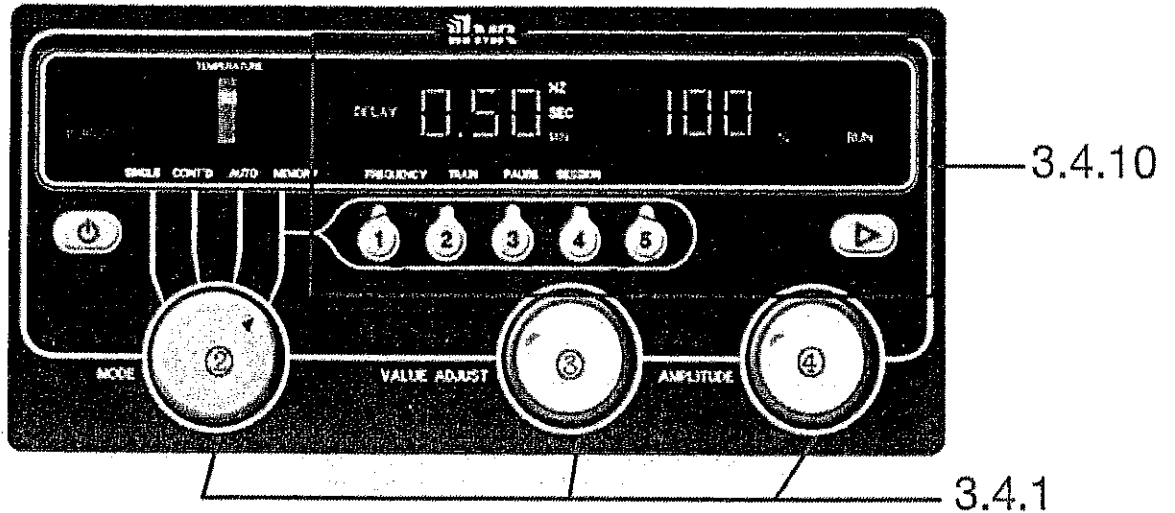3. 2. Function descriptions of control panel's buttons & dials

* Descriptions

- LCD panel shows various values according to each modes.


Coil temperature indicator: this indicates the temperature coil temperature when equipment work. There are five-segments from green to yellow and red. Generally, it indicates 2nd or 3rd segment (green) only because main body has cooling system using distilled water. If MS's cooling system has any kind of problem, it indicates to red color and MS will be stop immediately.

Segment	Temperature range
1st green segment	at less than 30°C
2nd green segment	at coil temperature from 30°C up to 34°C
3rd green segment	at coil temperature from 34°C up to 38°C
Yellow segment	at coil temperature from 38°C up to 41°C
Red segment	if the coil temperature exceeds 41°C, at that the button "RUN" and stimulus delivery commands from all the external devices are blocked.

Handwritten signature and stamp. The stamp is circular with the text 'CHINNAORIT CO., LTD.' around the perimeter and a star in the center.

3. Parts and functions

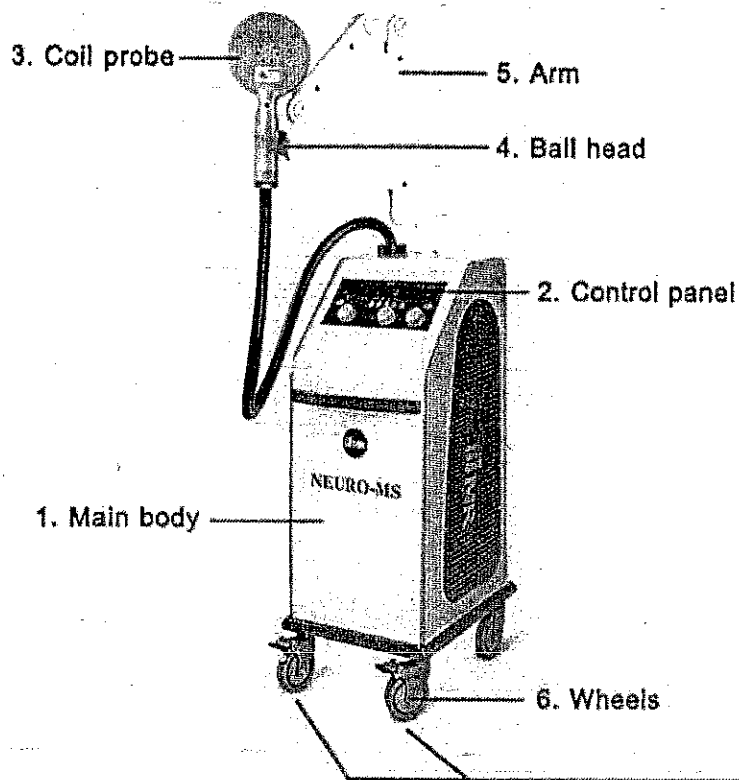3. 1. Parts name & description of main body

3.4.11

No.	Part name	Description
1	Main body	Part including power supplier, charge-discharge part, chiller for cooling and control panel, etc.
2	Control panel	Part controlling stimulation & various functions - frequency, intensity, session time and modes.
3	Coil probe	Electromagnetic pulse inducing part
4	Ball head	Part loosing & holding coil probe neck
5	Arm	Part controlling treatment position with point joint
6	Wheel	For movement in places

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Sonopuls 492

ELECTRICAL SPECIFICATIONS



Power supply	100 – 240 V ± 10% / 50 / 60 Hz
Dimensions (WxDxH)	
Device	24 x 32 x 12 cm
Device with base	24 x 30.5 x 18.2 cm
Device with Vacotron	24 x 30.5 x 21.6 cm
Weight	2 kg
Weight, including the optional battery	3 kg

4.1.1.1

Ultrasound frequencies	1 en 3 MHz 4.1.1
Ultrasound	continuous and pulsed } 4.1.2
Duty cycles	5, 10, 20, 33, 50, 80%
Pulse frequency	16 Hz, 48 Hz en 100 Hz
Intensity	0-2 W/cm ² continuous, 0-3 W/cm ² pulsed ⁽¹⁾ 4.1.3
Number of US connections	2
Multi-frequency treatment head large	5 cm ²
Multi-frequency treatment head small	0.8 cm ²
Pre-programmed treatment suggestions	25 - Evidence Based } 4.1.7
Programmable positions	300

This is a medical device.

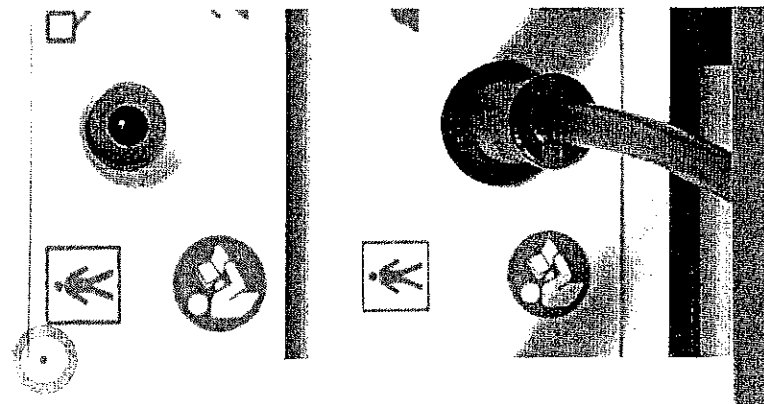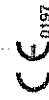It complies with all applicable medical regulations. For more details, please consult the instructions for use (downloadable via www.enraf-nonus.com).

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
ELECTROACUPUNCTURE

Electrotherapy 4.2

Current channels	2 (100% independent)
Current types	26 (per channel) 4.2.1
Pre-programmed treatment suggestions	42 - Evidence Based } 4.1.7
Programmable positions	300

Cardiovascular therapy

Pre-programmed treatment suggestions	2 - Evidence Based } 4.1.7
Programmable positions	300

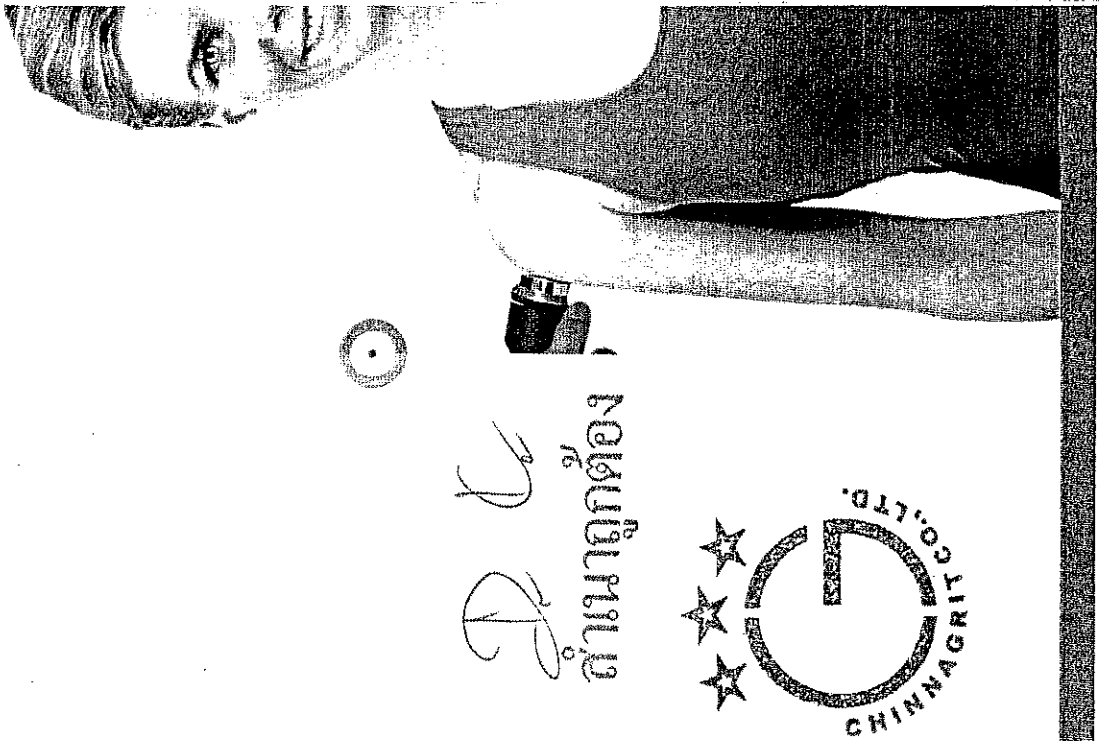Ultrasound parameters

Ultrasound Frequency, expressed in MHz, is the frequency of the ultrasound waves. The ultrasound frequency determines the penetration depth, which has the largest value at 1 MHz. The ultrasound frequency can be set at 1 MHz or 3 MHz.

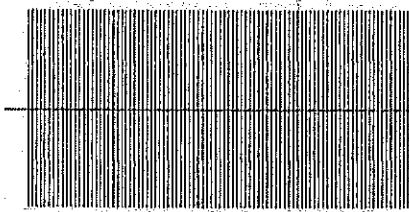
Duty Cycle, expressed in %, defines the ratio of the pulse duration to the pulse repetition time. Ultrasound can be applied in pulsed or in continuous mode. When the Duty Cycle is set to 100%, the apparatus operates in continuous mode.

Effective Radiation Area (ERA) expressed in cm^2 , defines the cross-sectional area of the ultrasound beam (See technical specifications for details). The Effective Radiation Area is fixed and defined by the size of the ultrasound applicator.

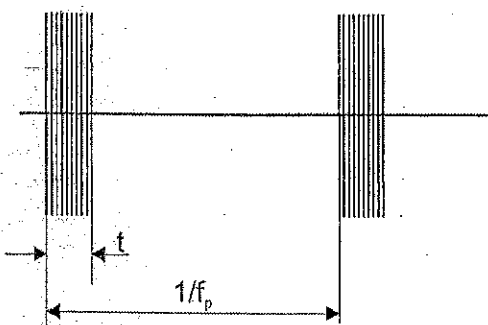
Ultrasound Power is the ultrasound output expressed in W. The ultrasound output display can be toggled between W and W/cm^2 . In pulsed mode the power during the pulse is displayed. The time averaged power can be obtained by multiplying this value with the Duty Cycle.

Ultrasound Amplitude, expressed in W/cm^2 , is the quotient of Ultrasound Power and Effective Radiation Area. The ultrasound output display can be toggled between W and W/cm^2 . In pulsed mode the Amplitude during the pulse is displayed. The time-averaged Amplitude can be obtained by multiplying this value by the Duty Cycle.

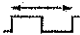
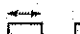
Continuous ultrasound

Pulsed ultrasound

Explanation of symbols:

-  f Ultrasonic radiation with an acoustic working frequency of 1 or 3 MHz
-  f_p Pulse repetition rate
-  dc Duty cycle
-  t Pulse duration

Ultrasound parameters:

- f Acoustic working frequency: 1MHz or 3MHz
- f_p Pulse repetition rate: 16, 48 and 100 Hz
- dc Duty cycle: 5 – 80 %
- t Pulse duration: 0.5 – 8 ms, set by duty cycle
- RTPA 20 – 1.25

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10 General instructions

Ultrasound therapy

- ⚠ During treatment, the patient may not feel unpleasant sensations amounting to pain. A mild sensation of excitation is permissible.
- ⚠ If, as result of treatment, headache, vertigo, fatigue and/or other (autonomic nervous) reactions develop, subsequent treatment should be given at a lower intensity.
- ⚠ With continuous and pulsed ultrasound at high intensity a sensation of heat may be felt. Only a mild sensation of warmth is acceptable.

Before treatment

Please make sure you have read and understood the content of this manual before you start treatment.

Check the patient for any possible contra-indications.

Test the thermal sensitivity of the treatment area.

The skin of the area in question is cleaned (removal of grease) with soap or 70 % alcohol to permit optimal transmission of the ultrasound.

Starting treatment

Position the treatment head (do not forget to apply gel on the patient).

When adequate contact is made, the timer will start.

When the contact between the treatment head and the patient is not adequate, LED ring (contact control indicator) in the treatment head will turn on.

During treatment

The treatment head is kept in slow continuous motion, also for the semi-static method.

During treatment, the displayed ultrasound amplitude can vary around the set value, caused by fluctuations in acoustical coupling. The patient is regularly asked to report any sensations felt. If necessary, the treatment is modified; 4.1.4

The intensity can be reduced or a switch is made from continuous to pulsed ultrasound.

If there are indications of poor transfer of the ultrasound energy the contact medium can be reviewed if necessary, add more contact gel or spread it with the ultrasound treatment head.

Important!

In order to ensure efficient transfer of energy, a contact medium is required between the treatment head and the body. Air reflects virtually all of the ultrasound energy. The best medium for the transfer of ultrasound energy is a gel.

- For preference, use Enraf-Nonius Contact-Gel® , as this allows the excellent characteristics of the treatment heads to be used to their full advantage.
- The gel should be applied to the part of the body to be treated and then spread out with the treatment head.

Ending treatment

Treatment can be stopped by taking off the treatment head from the patient and setting the timer to zero. Treatment stops automatically when the treatment time has elapsed.

After treatment

The patient's skin and the ultrasound treatment head are cleaned with a towel or tissue.

The ultrasound treatment head must be cleaned as described in chapter 14.

The expected effects are checked (e.g. pain, circulation, and mobility).

The patient is asked to comment subsequently on any reactions that may occur.

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Current waveforms – Pain management

For pain management, the following current waveforms are recommended.

- 4-Polar Interferential Currents
- Biphasic Pulsed Currents (TENS)
- Premodulated
- Micro Current
- High Voltage
- Diadynamic Currents
- Galvanic Current
- Träbert, 2 – 5 Current

Current waveforms – Muscle stimulation

For muscle stimulation, the following current waveforms are recommended.

- Biphasic Pulsed Currents (TENS)
 - Asymmetrical and Alternating asymmetrical
 - Symmetrical
- Premodulated
- Russian Stimulation
- High Voltage
- Faradic Current
 - Faradic Rectangular or Triangular pulsed current

These waveforms are often applied in combination with a surge program, which consists of a sequence of exercise and rest periods. Two options are available here:

- Reciprocal application, where stimulation alternates between agonists and antagonists. This is accomplished through asynchronous stimulation over two current channels with an appropriate delay between the two channels.
- Co-contract application, where two channels operate synchronously to co-contract agonist and antagonist or different sections of a larger muscle group.

Current waveforms – Combination therapy 4.1.5

Combination therapy is available with all current waveforms, but limited to channel 2.

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Vacuum electrodes

There is a choice of large and small electrodes. The areas of the electrodes correspond to those of the 4 x 6 cm and 6 x 8 cm flexible rubber electrodes. The vacuum electrodes are sufficiently flexible to ensure optimum contact with the skin, but rigid enough to prevent any changes in the contour of the part being treated, allowing full advantage to be taken of the massage effect of the pulsed vacuum.

Keep the sponge pads well moistened (close to dripping when lightly squeezed) during treatment. Remoisten when treatment exceeds 10 minutes. After use clean the sponge pads as described under *Cleaning and Disinfection*.

Self-adhesive electrodes

Self-adhesive electrodes have higher series impedance than flexible rubber electrodes. This can cause the stimulator to terminate treatment at higher current amplitudes. When this occurs, it is recommended to continue the treatment with flexible rubber electrodes, combined with properly moistened sponge pads.

Self-adhesive electrodes are not recommended for use with currents that contain a DC component.

Ultrasound therapy**Contact control 4.1.6**

The ultrasound head has a contact control function that suspends treatment when the acoustical contact with the body drops below a certain level. The indicator light on the ultrasound head is turned on to signal this situation, the ultrasound channel status icon changes to pause and the treatment is resumed at the set Amplitude.

Amplitude display will start blinking and the treatment timer will stop counting down. During this situation, the applicator emits a small amount of energy to sense restoration of acoustical contact. You may experience this when the ultrasound head only partially contacts the body. When contact restoration is sensed the treatment is resumed at the set Amplitude.

Note: The contact control function does not work at Amplitudes below 0.2 W/cm².

The contact medium

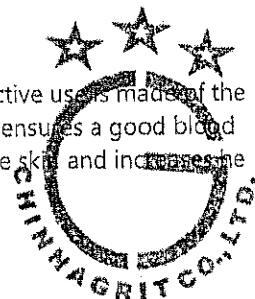
- ⚠ Never apply the gel directly to the ultrasound applicator. The applicator will register this as acoustical contact and may emit ultrasound energy, which could damage the applicator. If the body surface is very irregular, making it difficult to obtain good contact between the ultrasound applicator and the body, or if direct contact must be avoided (e.g. due to pain), the affected area may be treated under water (subaqual method). The water should be degassed (by previous boiling) in order to prevent air bubbles arising on the ultrasound applicator and the body.

To ensure efficient transfer of energy, a contact medium is required between the ultrasound applicator and the body. Air causes virtually total reflection of the ultrasound energy. The best medium for the transfer of ultrasound energy is a gel.

The gel should be applied to the part of the body to be treated and then spread out with the ultrasound applicator.

Vacuum

Vacuum electrodes make good contact with the skin, which means that effective use is made of the whole electrode area. The massage effect resulting from the pulsed vacuum ensures a good blood flow through the skin under the electrodes. This reduces the resistance of the skin and increases the effectiveness of the stimulation current.

Amplitude modulation	Duty cycle	Pulse duration	RTPA	Amplitude modulation	Duty cycle	Pulse duration	RTPA
16Hz	5%	3.1ms	20	48Hz	50%	10.4ms	2
16Hz	10%	6.3ms	10	48Hz	80%	16.7ms	1.25
16Hz	20%	12.5ms	5	48Hz	100%*	20.8ms	1
16Hz	33%	20.6ms	3	100Hz	5%	0.5ms	20
16Hz	50%	31.3ms	2	100Hz	10%	1ms	10
16Hz	80%	50ms	1.25	100Hz	20%	2ms	5
16Hz	100%*	62.5ms	1	100Hz	33%	3.3ms	3.33
48Hz	5%	1ms	20	100Hz	50%	5ms	2
48Hz	10%	2.1ms	10	100Hz	80%	8ms	1.25
48Hz	20%	4.2ms	5	100Hz	100%*	10ms	1
48Hz	33%	6.9ms	3.33				

* = continuous mode

Generator

Peak output Amplitude:

- Duty cycle 5 – 50 % : 0 – 3 W/cm²
- Duty cycle 80 % : 0 – 2.5 W/cm²
- Duty cycle 100 % : 0 – 2 W/cm² (continuous wave)

Peak output power for 5 cm² applicator:

- Duty cycle 5 – 50 % : 0 – 15 W
- Duty cycle 80 % : 0 – 12 W
- Duty cycle 100 % : 0 – 10 W (continuous wave)

Peak output power for 0.8 cm² applicator:

- Duty cycle 5 – 50 % : 0 – 2.4 W
- Duty cycle 80 % : 0 – 2 W
- Duty cycle 100 % : 0 – 1.6 W (continuous wave)

Output meter uncertainty : ± 20 % for any output above 10 % of maximum

- Pulse frequency : 16, 48 and 100 Hz ± 1 %
- Duty cycle : 5 – 80 % and 100 % (100 % is continuous wave)
- Pulse duration : 0.5 – 8 ms ± 10 % (set by duty cycle)
- Temporal Peak to Average Ratio (RTPA) : 20 – 1.25 ± 5 % (set by duty cycle)

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Stimulator output parameters**Electrotherapy general 4.2**

Channels	: 2
Output characteristics	: Constant Current (CC) or Constant Voltage (CV), except for High Voltage (CV) and Microcurrent (CC).
Current amplitude range	: Depending on current waveform
Current amplitude resolution	: 0.2 mA
Treatment timer	: 0 – 60 minutes 4.2.1.1.1
Polarity reversion direct currents	: manual

The maximum current amplitude within the specification is achieved up to a load of 500 Ω (CC).

Surge program

With some current waveforms a surge program is available.

The parameters and their range are as follows:

Ramp up time	: 0 – 9 s, in steps of 1s
Hold time	: 0 – 60 s, in steps of 1 second
Ramp down time	: 0 – 9 s, in steps of 1s
Interval time	: 0 – 120 s, in steps of 1 second
Delay time	: 0.1 – 80 s; below 10s in steps of 0.1s, otherwise in steps of 1s

4.2.1**Interferential 4 polar**

Carrier Frequency	: 2, 2.5, 3, 3.5, 4, 5, 6, 7, 8, 9, 10 kHz
Beat Frequency (AMF)	: 0 – 200 Hz in steps of 1 Hz
Frequency Modulation (spectrum)	: 0 – 180 Hz in steps of 1 Hz
Modulation program	: 1/1, 6/6, 12/12, 1/30/1/30 s
Amplitude	: 0 – 100 mA

Isoplanar vector

Carrier Frequency	: 2, 2.5, 3, 3.5, 4, 5, 6, 7, 8, 9, 10 kHz
Beat Frequency (AMF)	: 0 – 200 Hz in steps of 1 Hz
Frequency Modulation (spectrum)	: 0 – 180 Hz in steps of 1 Hz
Modulation program	: 1/1, 6/6, 12/12, 1/30/1/30 s
Amplitude	: 0 – 100 mA

4.2.1.1

Dipole Vector Automatic

Carrier Frequency	: 2, 2.5, 3, 3.5, 4, 5, 6, 7, 8, 9, 10 kHz
Beat Frequency (AMF)	: 0 – 200 Hz in steps of 1 Hz
Amplitude	: 0 – 100 mA
Rotation time	: 1 – 10 s in steps of 1 s

Dipole Vector Manual

Carrier Frequency	: 2, 2.5, 3, 3.5, 4, 5, 6, 7, 8, 9, 10 kHz
Beat Frequency (AMF)	: 0 – 200 Hz in steps of 1 Hz
Frequency Modulation (spectrum)	: 0 – 180 Hz in steps of 1 Hz
Modulation Program	: 1/1, 6/6, 12/12, 1/30/1/30 s
Amplitude	: 0 – 100 mA
Resolution vector	: 2.25° per step (160 steps over 360°)

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4.2.1

Pulse Frequency	: 1 – 200 Hz, in steps of 1 Hz
Frequency Modulation	: 0 – 180 Hz, in steps of 1 Hz
Modulation Program	: 1/1, 6/6, 12/12, 1/30/1/30 s
Surge Program	: Yes
Amplitude	: 0 – 140 mA

Asymmetrical Alternating

Phase Duration	: 10 – 400 μ s in steps of 5 μ s
Pulse Frequency	: 1 – 200 Hz, in steps of 1 Hz
Frequency Modulation	: 0 – 180 Hz, in steps of 1 Hz
Modulation Program	: 1/1, 6/6, 12/12, 1/30/1/30 s
Surge Program	: Yes
Amplitude	: 0 – 140 mA

Burst Asymmetrical

Phase Duration	: 10 – 400 μ s in steps of 5 μ s
Pulse Frequency	: 1 – 200 Hz, in steps of 1 Hz
Burst Frequency	: 1 – 9 Hz, in steps of 1 Hz
Amplitude	: 0 – 140 mA

Burst Asymmetrical Alternating

Phase Duration	: 10 – 400 μ s in steps of 5 μ s
Pulse Frequency	: 1 – 200 Hz, in steps of 1 Hz
Burst Frequency	: 1 – 9 Hz, in steps of 1 Hz
Amplitude	: 0 – 140 mA

Symmetrical 4.2.1.2

Phase Duration	: 10 – 400 μ s in steps of 5 μ s
Pulse Frequency	: 1 – 200 Hz, in steps of 1 Hz
Frequency Modulation (spectrum)	: 0 – 180 Hz in steps of 1 Hz
Modulation Program	: 1/1, 6/6, 12/12, 1/30/1/30 s
Surge Program	: Yes
Amplitude	: 0 – 140 mA

Burst Symmetrical

Phase Duration	: 10 – 400 μ s in steps of 5 μ s
Pulse Frequency	: 1 – 200 Hz, in steps of 1 Hz
Burst Frequency	: 1 – 9 Hz, in steps of 1 Hz
Amplitude	: 0 – 140 mA

Premodulated 4.2.1.1

Carrier Frequency	: 2, 2.5, 3, 3.5, 4, 5, 6, 7, 8, 9, 10 kHz
Beat Frequency	: 0 – 200 Hz in steps of 1 Hz
Frequency Modulation (spectrum)	: 0 – 180 Hz in steps of 1 Hz
Modulation Program	: 1/1, 6/6, 12/12, 1/30/1/30 s
Surge Program	: Yes
Amplitude	: 0 – 100 mA

Russian Stimulation 4.2.1.4

Carrier Frequency	: 2, 2.5, 3, 3.5, 4, 5, 6, 7, 8, 9, 10 kHz
Burst Frequency	: 0 – 100 Hz in steps of 1 Hz

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High Voltage (Twin Pulse) 4.2.1.5

Frequency : 1 – 200 Hz in steps of 1 Hz
 Frequency Modulation (spectrum) : 0 – 180 Hz in steps of 1 Hz, sum of Pulse frequency and Frequency Modulation cannot exceed 200 Hz
 Modulation Program : 1/1, 6/6, 12/12, 1/30/1/30 s
 Surge Program : Yes
 Polarity : Positive or Negative
 Amplitude : 0 – 500 Volt in steps of 1V

High Voltage Alternating (Twin Pulse)

Frequency : 1 – 200 Hz in steps of 1 Hz
 Frequency Modulation (spectrum) : 0 – 180 Hz in steps of 1 Hz, sum of Pulse frequency and Frequency Modulation cannot exceed 200 Hz
 Modulation Program : 1/1, 6/6, 12/12, 1/30/1/30 s
 Alternating Period : 10 – 100 seconds in steps of 10 s
 Ramp up, ramp down : 0.5 seconds
 Amplitude : 0 – 500 Volt in steps of 1 Volt

Micro Current 4.2.1.6

Frequency : 0 – 1000 Hz
 Polarity : Positive or Negative
 Surge Program : Yes
 Amplitude : 10 μ A – 1 mA in steps of 10 μ A

Micro Current Alternating

Frequency : 0 – 1000 Hz, 0 – 100 Hz in steps of 0.1 Hz, 10 – 100 Hz in steps of 1 Hz, 100 – 1000 Hz in steps of 10 Hz
 Alternating Period : 0.2 – 20 seconds, 0.2 – 1 s in steps of 0.1s, 1 – 20 s in steps of 1 s
 Ramp up, ramp down : 0 seconds
 Amplitude : 10 μ A – 1 mA in steps of 10 μ A

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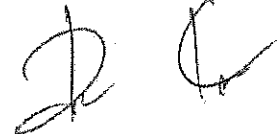
4.2.1

Faradic Rectangular Pulsed Current (ms) 4.2.1.8

Phase Duration : 0.02 – 1000 ms
 Phase Interval : 5 – 50 ms in steps of 5 ms, 50 – 100 in steps of 10 ms, 100 – 1000 ms in steps of 100 ms, 1 – 5 seconds in steps of 1 second
 Surge Program : Yes
 Polarity : Positive or Negative
 Amplitude : 0 – 80 mA

Faradic Triangular Pulsed Current (ms) 4.2.1.9

Phase Duration : 0.1 – 1000 ms
 Phase Interval : 5 – 50 ms in steps of 5 ms, 50 – 100 in steps of 10 ms, 100 – 1000 ms in steps of 100 ms, 1 – 5 seconds in steps of 1 second
 Surge program : Yes
 Polarity : Positive or Negative
 Amplitude : 0 – 80 mA

Faradic Rectangular Pulsed Current (Hz)

Phase Duration : 0.02 – 1000 ms
 Pulse Frequency : 0.2 – 1 Hz in steps of 0.1 Hz, 1 – 200 in steps of 1 Hz
 Surge Program : Yes
 Polarity : Positive or Negative
 Amplitude : 0 – 80 mA

Faradic Triangular Pulsed Current (Hz)

Phase Duration : 0.1 – 1000 ms
 Pulse Frequency : 0.2 – 1 Hz in steps of 0.1 Hz, 1 – 200 in steps of 1 Hz
 Surge program : Yes
 Polarity : Positive or Negative
 Amplitude : 0 – 80 mA

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
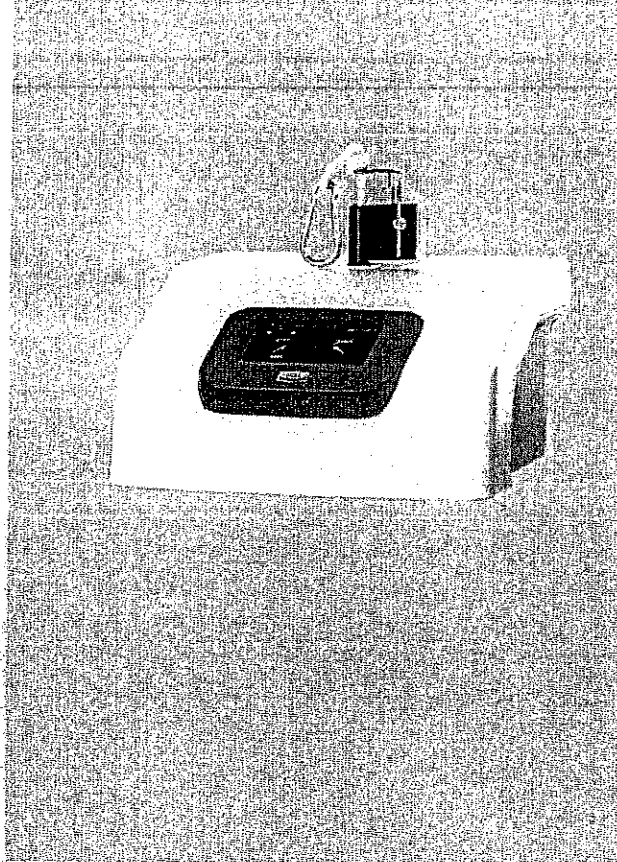
1671901 ELTRAC 471

TECHNICAL SPECIFICATIONS


Form of therapy:	Static, Intermittent, Pro- and Regressive, 5.3
Traction force:	15 - 900 N / 1.5 - 90 kg / 3.4 - 202 Lbs 5.4, 5.5
<u>Traction force / base force hold time:</u>	1 sec. - 1 hour (1 sec. intervals) 5.7
Repeats (intermittent and cyclic):	1 - 100
Steps hold time (pro- and regressive):	1 sec.- 5 min. (1 sec. intervals)
Steps (pro- and regressive):	1 - 25
<u>Speed (change of traction force):</u>	1 - 50 N/sec. (1 N intervals) } 5.6 0.1 - 5.0 kg/sec. 0.2 - 11.2 Lbs/sec.
Treatment time:	1 sec. - 150 min.
Programmable favorites:	100+
Clinical protocols:	Yes (3)
Sequential programming:	Yes (up to 5 forms of therapy)
<u>Safety control for traction force above:</u>	200 N / 20 kg / 45.2 Lbs } 5.8
Touch screen:	4.3 inch, full colour, capacitive
Mains voltage:	100 - 240 V (50/60 Hz)
Max. power consumption:	50 VA
Dimensions:	30 x 34 x 22 cm
Weight:	10 kg

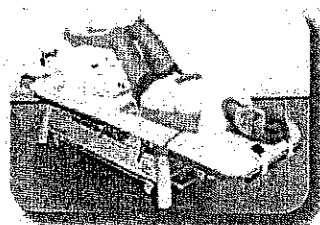
ENRAF-NONIUS
P.O. Box 12080
3004 GB Rotterdam
The Netherlands
T: +31-(0)10 - 203 06 00
E: info@enraf-nonius.nl
www.enraf-nonius.com

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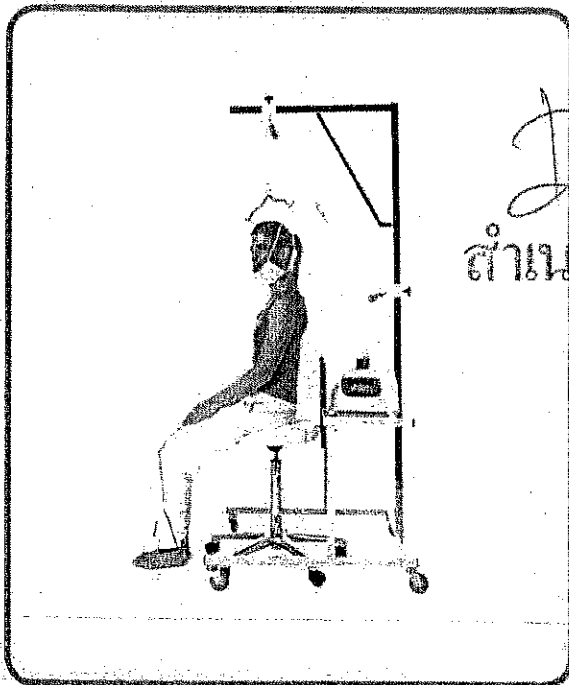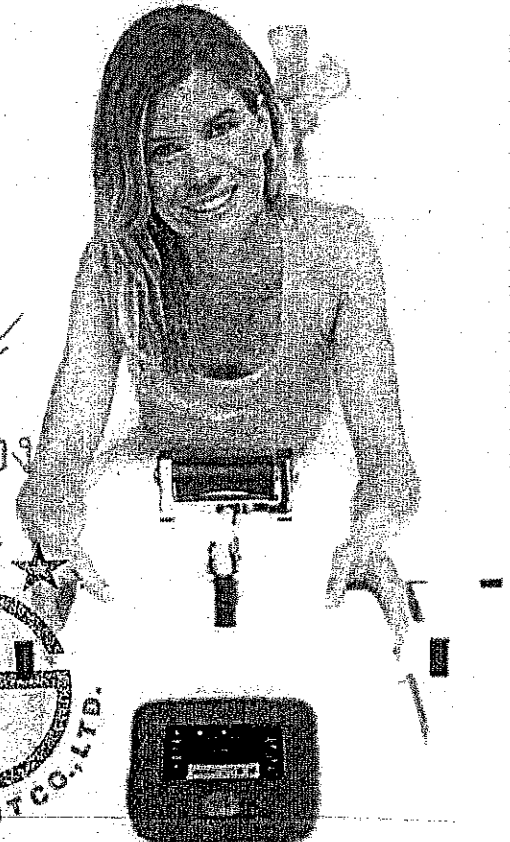ELTRAC 471

ACCURATE, EASY-TO-USE, REAL-TIME AND SAFE

MADE BY ENRAF-NONIUS

Made by Enraf-Nonius and, therefore, safe, reliable and durable. A device made by Enraf-Nonius, THE supplier for the physiotherapist worldwide. Where quality, reliability and durability come as standard.

TRACTION COUCH WITH EASY FIXATION

With fixation rails on both sides (type 916 and 926) as well as on the head section, the Manumed Traction offers a possibility to fixate the patient in any possible way. Traction belts and various Velcro straps can easily be attached. Fixation is an easy task on a Manumed Traction couch.

5.2 CRYSTAL CLEAR TFT COLOUR SCREEN

A crystal clear, full colour and scratch resistant screen that allows full visibility on all parameters from all viewing angles.

PROGRAMMABLE

In addition to the standard treatment protocols, you can also save your personal protocols (including sequential applications) based on your own guidelines. Every conceivable combination - static, pro- and regressive, cyclic or intermittent can be stored as a personal favourite in this way.

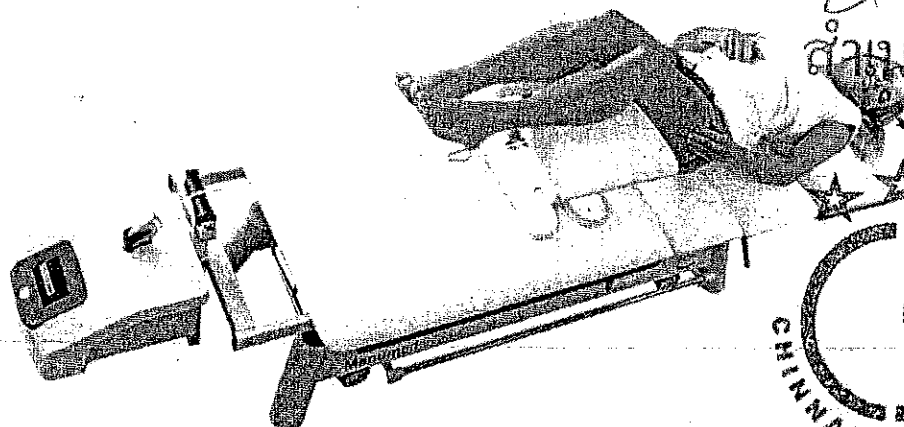
5.2 TOUCH SCREEN USER INTERFACE

Easy to use by means of a clear menu via a touch screen interface. Extremely intuitive and as easy as a state-of-the-art smart phone.


FLEXISTOOL FOR AN IDEAL STARTING POSITION DURING LUMBAR TRACTION

The high-low adjustable flexistool renders it possible to exercise traction with an ideal starting position in which the hip-angle can be bent from 45 to 90 degrees.

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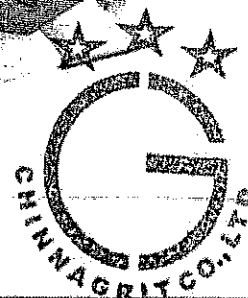- Mechanical max. force switch
- Traction cord
- Remote control connector
- Patient switch connector

Electronic overview

All essential electronic hardware of the Eltrac 471 unit is found on 2 PC Boards.

The Control PC Board:

5.1
Application software, necessary for operation of the unit, runs at the processor section. This software is found on a micro SD-Card. The processor section controls and checks the set treatment parameters. It also controls the LCD display and touch screen interface circuits.

The main PC Board circuits:

- The amplifier circuit amplifies the electronic signal from the strain gauges. These strain gauges are found in a so called "Wheatstone bridge" circuit which is very sensitive to changes of the output force of the unit. If the strain gauges are stretched the resistance changes. The force is measured continuously and fed back to the Control PC Board. If the measured force is different from the set value it will be corrected.
- The solenoid driver circuit operates the solenoid clutch. The solenoid is connected to an "intermediate gear clutch" construction, see mechanical overview below.
- A motor driver circuit drives the motor / reduction gearbox assembly. A mechanical max. force switch is also found in series of the driver circuit. It prevents the output force from getting too high. If the force gets too high the switch opens and the motor stops immediately. The max. force switch acts at approx. 1000N.

Mechanical overview

- The winding drum runs free as long as no treatment is started. The spring loaded drum pulls the traction cord into the unit. It is possible to pull the cord out approx. 80cm. to adapt the length of the cord required for treatment.
- The motor / gearbox assembly drives the winding drum. Depending on the rotation direction of the motor the output force is increased or decreased.
- When a treatment is started the solenoid is activated. As long as the solenoid is activated an intermediate gear "connects" the motor / gearbox to the winding drum.
- The traction cord is wound on the drum during treatment. It runs through the strain gauge assembly. The strain gauge shaft bends depending on the force applied. At the shaft two strain gauges are "glued" which are connected to the Main PC Board.
- The mechanical max. force switch is mounted on the chassis. The switching force can be adjusted, for details see the adjustment procedures at chapter 7.

4. Quick reference / selftest / functional test

Quick reference:

Figure 4 shows a picture of the Eltrac 471 unit.

- | | |
|--------|--|
| Item ① | Top cover with LCD display |
| item ② | LCD-Touch screen |
| item ③ | Stop switch connection |
| item ④ | Remote control connection |
| item ⑤ | Traction cord |
| item ⑥ | Service slider (for easy removal of the top cover) |
| item ⑦ | Mains supply connection |

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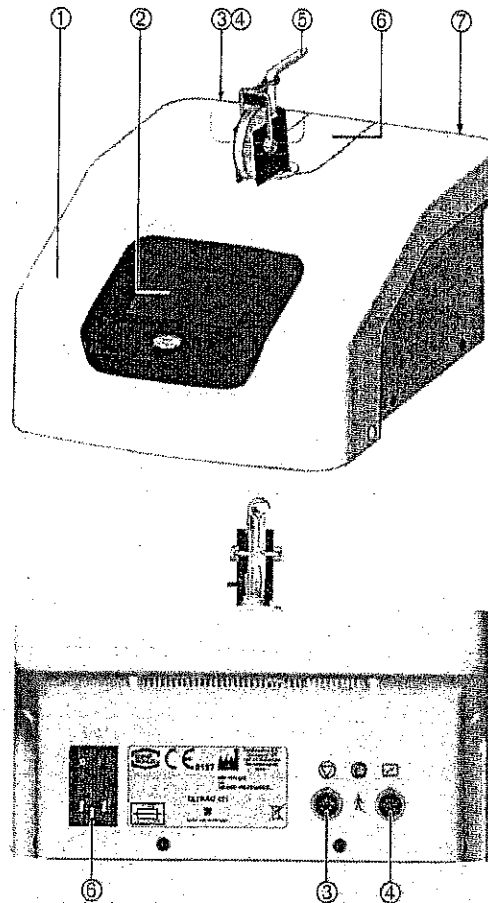
Figure 4 Eltrac 471 unit front and rear view.

4.1 **Selftest: 5.1**

After switching On the mains switch (refer to figure 4, item [6]) the unit runs a selftest. When the selftest is passed the [Home] menu is displayed, see figure 5 and a beep is heard. It is assumed that when the selftest sequence is passed, all functions of the Eltrac 471 unit are OK. If the unit does not switch On, does not enter the user mode or displays an error code, see the trouble shooting and error code list.

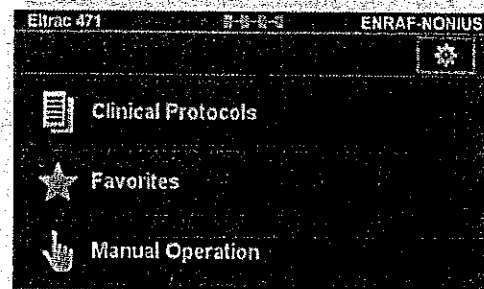
Figure 5 Eltrac 471 Home screen.

4.2 **Unit settings (System settings menu)**

Changes to the default system settings are made at the System Settings menu, enter the [Home] menu. Press the [toothed wheel] to enter the [System Settings] menu, see figure 5. For changing the (default) unit settings see the user manual for detailed information. Figure 6 shows the system settings menu, page 1. The system settings menu has a total of 3 pages.


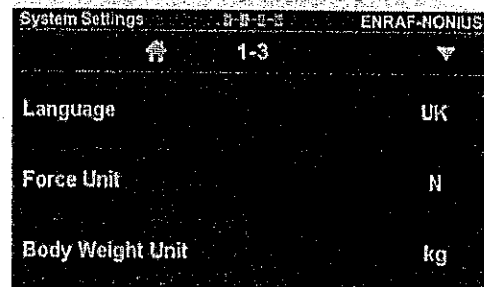
Figure 6 System settings screen.

At the [System settings] menu, the following options are found:

8.2.5 Manual Operation


Form of therapy

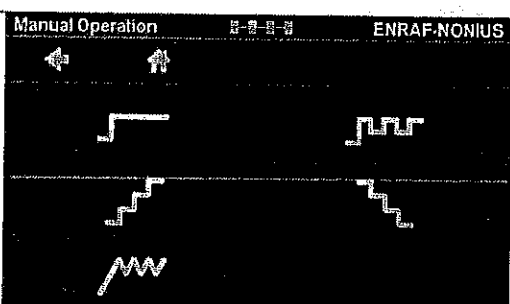After selecting Manual Operation in the main menu, the possible forms of therapy appear.
Depending on the selected form of therapy, you will be prompted to enter the required parameters.

Setting of parameters

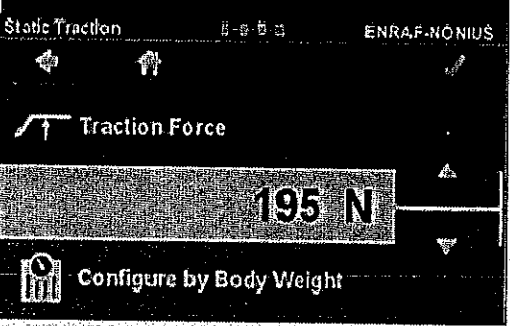
Use the arrow keys on the screen to adjust the parameter.
Use the tick on the navigation bar to confirm the set value.
The screen for setting the next parameter appears.
You can use the back key on the navigation bar to return to the previous parameter.

Setting with body weight


You have the option to set the body weight and a percentage to determine the required power. } 5.4
To do this, press the "Setting with body weight" key at the bottom of the screen.
Set the patient's body weight. In the next screen you can insert the percentage of the body weight.

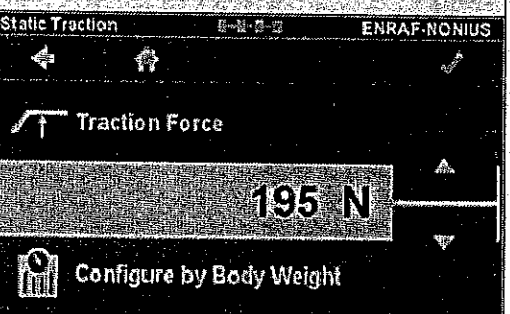(The desired tractive power will be calculated for you. The absolute value will be shown when you are in the treatment screen)

Speed

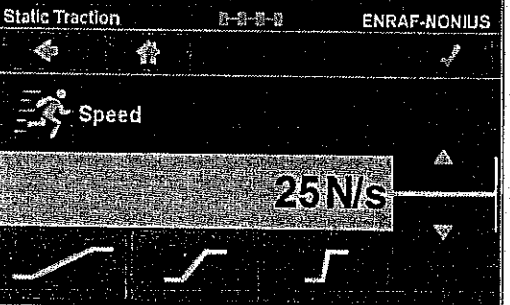The speed with which the tractive power changes can be set very accurately using the arrow keys.
Moreover you can use the three pictograms at the bottom of the screen: slow, medium and fast.



When all required parameters have been set, you will arrive at the treatment screen.

8.2.6 Treatment screen

Screen lay-out


At the right-hand side of the treatment screen you will be able to quickly view the main parameter settings.
In the middle of the screen the treatment setting will be depicted in a diagram.
On the left-hand side of the screen you see a symbol for the therapy form setting. In the case of a sequential programme you will find the sequence of the programmed forms of therapy from top to bottom.

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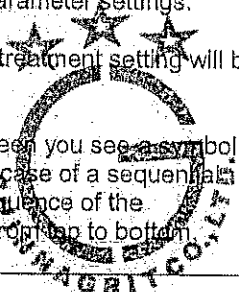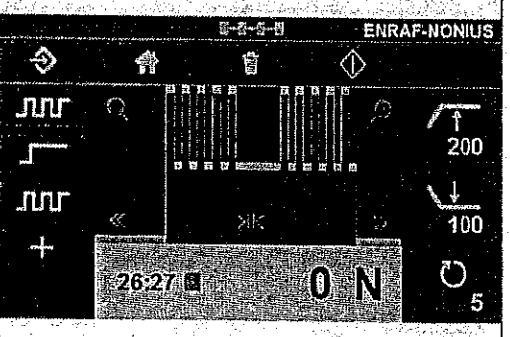Manumed Electric Traction treatment couch

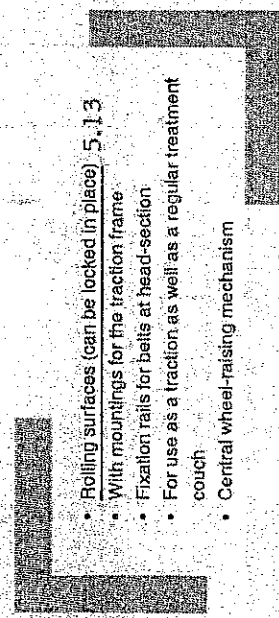
For the application of lumbar and cervical traction therapy there is the Manumed Traction couch. The roller-mounted surface prevents friction below the hips and legs, so that the full traction force set is actually applied in the vertebral treatment. The Manumed Traction is equipped with electrically adjustable working height. The variable-height traction couch also doubles as an ideal treatment couch, because the rolling sections can be locked in place.


- Rolling surfaces (can be locked in place) 5.13
- With mountings for the traction frame
- Fixation rails for belts at head-section
- For use as a traction as well as a regular treatment couch
- Central wheel-raising mechanism

Accessories Manumed Traction couch

3445021	Toracic belt for Manumed Traction	<input checked="" type="checkbox"/>
3445022	Palvic belt for Manumed Traction	<input checked="" type="checkbox"/>
3445036	Traction frame 430 mm width for type 926	<input checked="" type="checkbox"/>
3xxx014	FlexiSoo with separate height adjustment	<input checked="" type="checkbox"/>

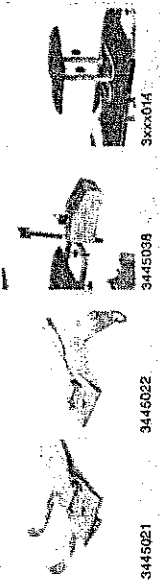
3445021 3445022 3445036 3xxx014

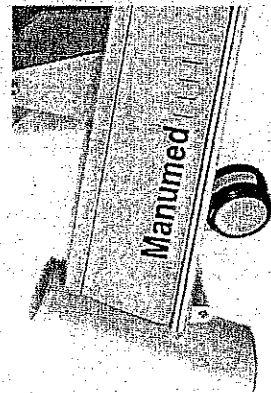Optional: All-around operation

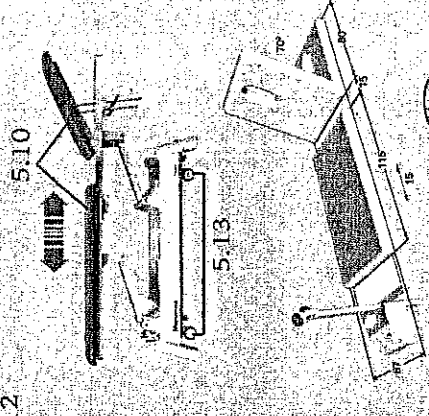MANUMED TRACTION HEIGHT ADJUSTABLE (ELECTRIC)


FRAME COLOURS

URPHOLSTERY COLOURS	012	013	004	169	141	189
	155	131	019	139	127	185
	011	012	022	030	033	051

ENRAF-NONIUS INTERNATIONAL จำกัด

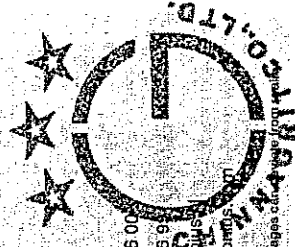Vareseweg 127
3047 AT Rotterdam
P.O. Box 12080
3004 GB Rotterdam
The Netherlands
Tel.: +31-(0)10 - 208 06 00
Fax: +31-(0)10 - 208 06 91
E-mail: info@enraf-nonus.nl
Website: www.enraf-nonus.nl

Technical applications and images call us at 010-2080600

TECHNICAL SPECIFICATIONS

Size tabletop (lxw): 195 x 67 cm; 5.11
Electric height adjustment: 46-97 cm; 5.12
Lifting time (min.-max.): approximately 18 sec.
Lifting capacity: 200 kg (optional 250 kg)
Force electrical motor: 8.000 N
Power supply: 120/230V AC 50/60 Hz
Current consumption: 2.0 A max.

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
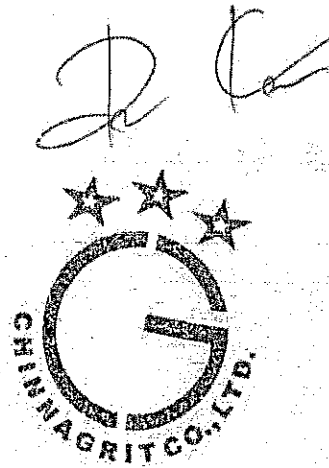
MANUMED EXERCISE COUCH

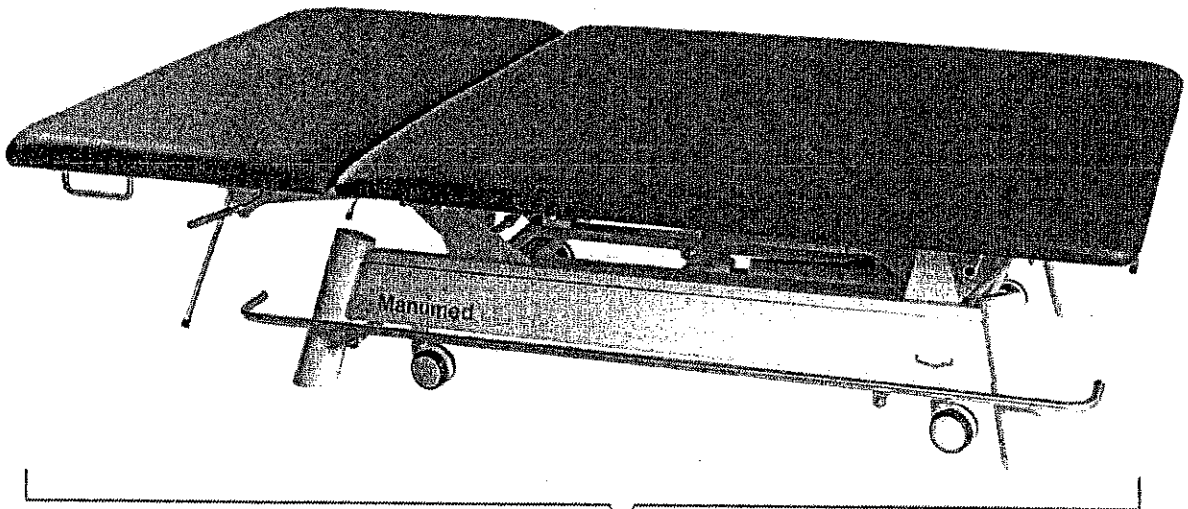Neurological exercise therapy with the Manumed Bobath. The large, extra wide treatment surface of 100 cm to 120 cm makes it easy to support and assist patients during therapeutic exercises. The Bobath and Vojta motion methods can be applied to perfection owing to the unmatched stability. A sturdy couch affording both patient and practitioner the most comfortable treatment positions. 9.1, 9.2

9.2

9.3

- Very large surface (100 or 120x200 cm)
- Wheel raising mechanism for easy movement of the couch 9.3
- Electric or hydraulic height adjustment
- The 2-section model can be used as a neurological exercise or treatment couch 9.1

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
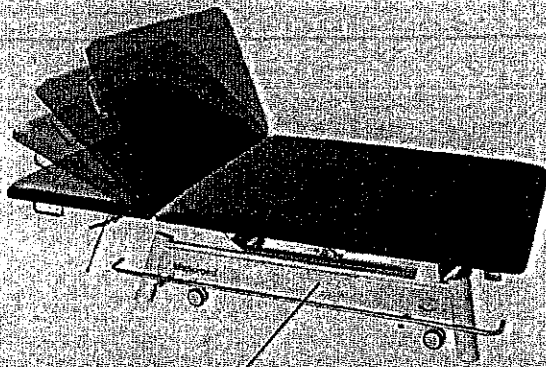
MANUMED EXERCISE COUCH

MANUMED EXERCISE 2-SECTION HEIGHT ADJUSTABLE

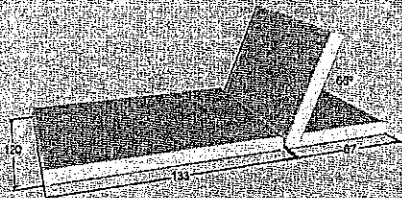9.4



Type

511 Manumed Exercise 2-section, hydraulic H/L

521 Manumed Exercise 2-section, electric H/L 9.6

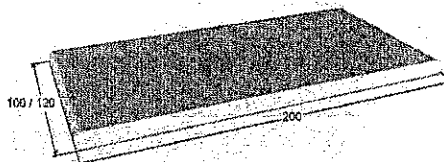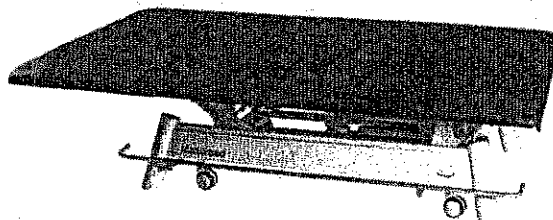MANUMED EXERCISE 1-SECTION HEIGHT ADJUSTABLE



Type

510 Manumed Exercise 1-section, hydraulic H/L

520 Manumed Exercise 1-section, electric H/L

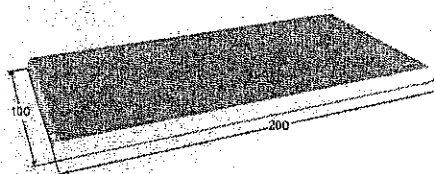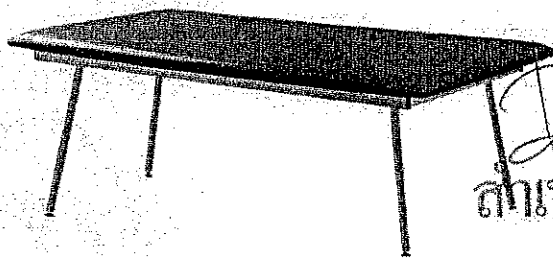MANUMED EXERCISE 1-SECTION FIXED HEIGHT

Type

500 Manumed Exercise 1-section, fixed height of 55 cm

TECHNICAL SPECIFICATIONS

Size tabletop (lxw)	200 x 100 cm for models 500-510 and 520 optional 120 cm width for 510 and 520 203 x 120 cm for models 511 and 521 9.5
Height adjustment	44-97 cm 9.6
Lifting time (min-max)	approximately 18 sec.
Lifting capacity	250 kg for type 520 and 521 (electric) 200 kg for type 510 and 511 (hydraulic) 135 kg for type 500 (fixed height)
Power supply	120/230VAC 50/60 Hz
Current consumption	2.0 A max

NOTE: ALL-AROUND OPERATION IS AN OPTION

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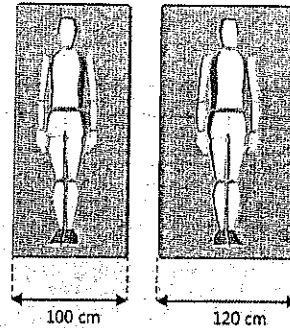
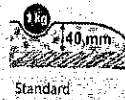
A

De C

(1) = NOTE: Please order options together with the couch because of modification of the couch during production

ENRAF-NONIUS

A PERFECT COUCH

Partner for Life

9.8

Extra luxurious /extra thick upholstery

Hydraulic or electric height adjustment

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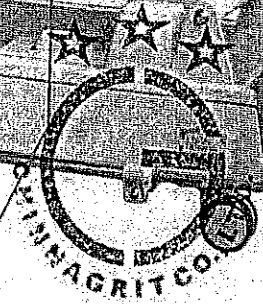
D & C

You will find an Enraf Nonius examination and treatment couch suitable for any treatment in any medical studio for any budget. Simple and sturdy or custom equipped and ready for specialized treatment. Whatever you choose, all the Enraf Nonius examination and treatment couches are of the highest quality, designed for the professional. We have a vast variety of accessories, rolls, cushions and stools available to complete your treatment couch of choice and your studio.

Download

The complete detailed brochure of the couch of your choice with all options can be downloaded from www.enrafnonius.com/couches

Technical data

- Scrub proof
- Resistant to saliva and perspiration
- Resistant to urine
- Resistant to blood
- Resistant to disinfections
- Resistant to coffee, tea, ketchup, red wine

9.8

Easy to clean and to maintain

- Clean with a mild soap and water
- Rinse with clear water and
- Dab dry with a soft cotton cloth

FREE OF PHTHALATES

Passed BS EN ISO 10993-5, 1999-11 Biological evaluation of medical devices part 5: Test for in vitro cytotoxicity. Passed BS EN ISO 10993-10, 2002 Biological evaluation of medical devices part 10: Test for irritation and delayed-type hypersensitivity

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The Certification Body of TÜV Rheinland LGA Products GmbH certifies that the organization has established and applies a quality management system for medical devices. Proof has been furnished that the requirements specified in the abovementioned standard are fulfilled. The quality management system is subject to yearly surveillance.

Report No.: 1099684-40
Effective date: 2022-03-01
Expiry date: 2025-02-28
Issue date: 2022-01-20

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Registration No.: SX 1274285-1

Organization: Enraf-Nonius B.V.
Vareseweg 127
3047 AT Rotterdam
Netherlands

The scope of certification also covers the following:

No.	Facility	Scope
/01	Enraf-Nonius B.V. Voltaweg 22 6101 XK ECHT Netherlands	Activities associated with the manufacturing, service and logistics

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
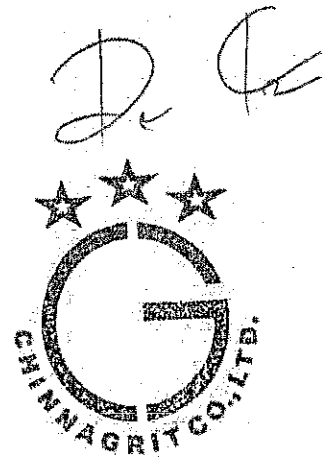
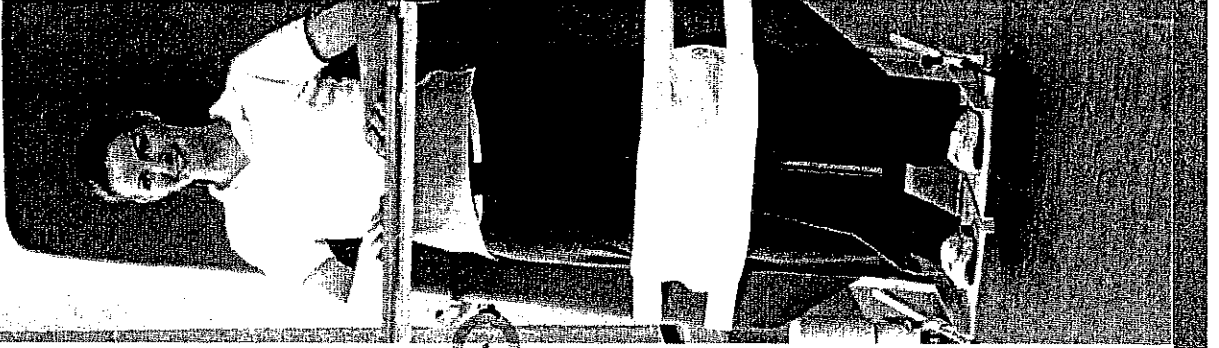
ORDERING DATA

Manumed Tilt 1-section

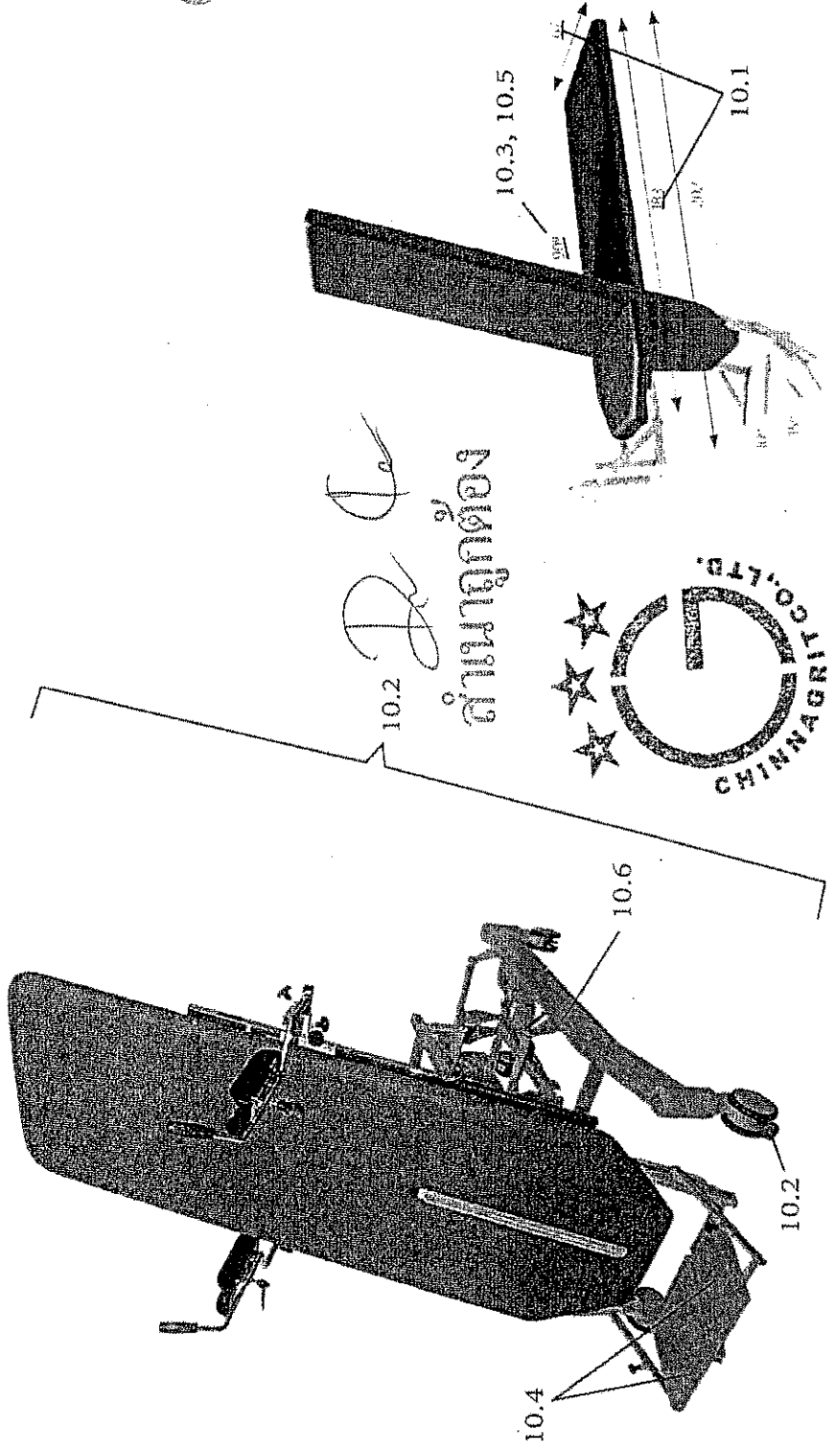
- Type
- 722 Manumed Tilt 1-section, electric H/L and electric tilting; No nose-hole
- 723 Manumed Tilt 1-section with leg fixation rails, electric H/L and electric tilting; No nose-hole


Manumed Tilt

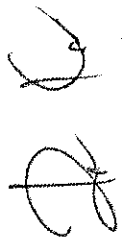- Tilts from 0 to 90° 10.3, 10.5
- Suitable for children
- Electrical height adjustment
- Large wheels for safe movement 10.2

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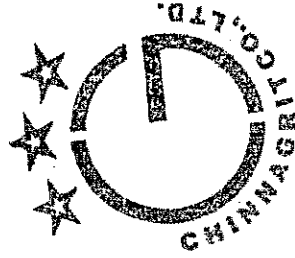Ergonomically placed handles

Central wheel raising mechanism

You will find an Enraf Nonius examination and treatment couch suitable for any treatment, for any medical studio, for any budget. Simple and sturdy or custom equipped and ready for specialized treatment. Whatever you choose, all the Enraf Nonius examination and treatment couches are of the highest quality, designed for the professional. We have a vast variety of accessories, rolls, cushions and stools available to complete your treatment couch of choice and your studio.

Download

The complete detailed brochure of the couch of your choice with all options can be downloaded from www.enraf-nonius.com/couches

Technical data

- Scrub proof
- **Resistant to saliva and perspiration**
- **Resistant to urine**
- **Resistant to blood**
- **Resistant to disinfections**
- Resistant to coffee, tea, ketchup, red wine

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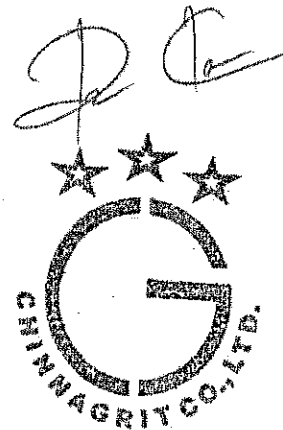
Curapuls 670

A-thermal and thermal

With its low-power settings, this device is exceptionally well suited to the treatment of acute conditions. Recent large-scale research has shown that this method is particularly well suited to the relief of post-operative pain and the promotion of post-operative wound healing.

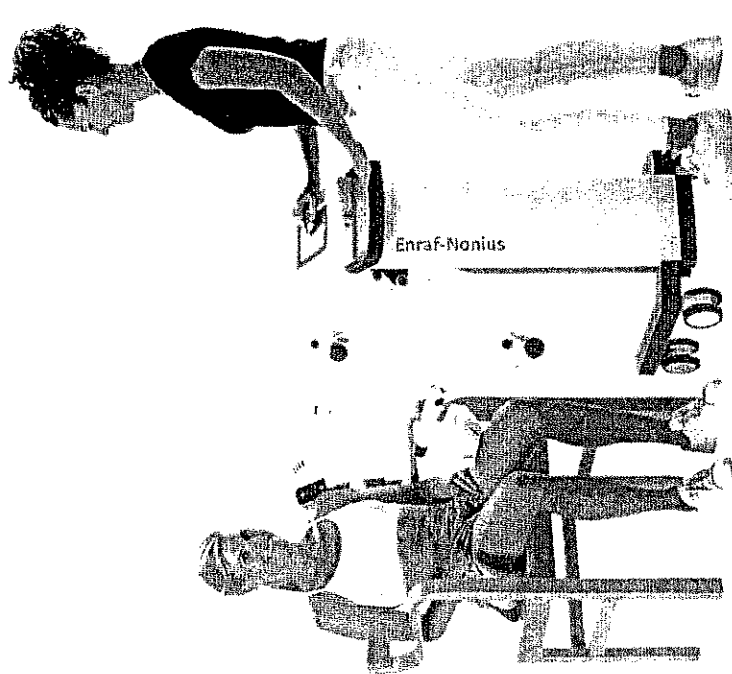
However, the device can also be used to achieve thermal effects, as well. Even these higher levels of power can be safely administered, and they are very effective in the relief of pain caused by chronic conditions such as osteoarthritis.

Easy to use and hands free

This therapy can be administered entirely hands free using the convenient arm, which can be fixed in the right position with just one button. This makes the device a clear improvement on labour-intensive therapies such as Thecar, which are generally administered to treat the same problems. The clever user interface and the touch screen also help therapists administer the therapy easily and without any effort. 13.1

A second arm can be affixed if necessary. Therapists can simultaneously use two circuplodes to approach the treatment area in a highly effective manner. Thanks to its two completely independent channels, the device can even be used to treat two areas at once, or even to treat two different patients at the same time.

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Curapuls 670

The therapy

The device transmits electromagnetic energy in the radio frequency spectrum. This therapy is known as PRFE (pulsed radio frequency energy) or PSWT (pulsed shortwave therapy).

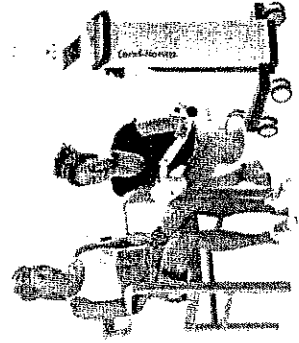
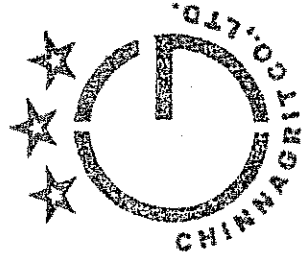
13.2

Curapuls 670 generates electromagnetic energy with a frequency of 27.12 MHz.

Due to the pulsed nature of the energy, the device can very briefly transmit high peak power Watts, followed by relatively long periods of inaction. As a result, the patient's tissue receives intense stimulation, although the therapy is very safe and comfortable for the patient.

Due to the nature of the electromagnetic energy, it is mainly absorbed by tissue with a low level of impedance, such as well-vascularised tissue (muscle tissue) or tissue featuring oedema or recent haematomas. Absorption of this energy promotes tissue recovery and improves circulation. Shortwave therapy is not very physically demanding and is effective very deep inside the body. The patient hardly experiences any thermal sensation, but the effect on his or her deep tissue is significant!

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
- High adjustable peak power (maximum 200 Watt)
- Pulse width and pulse frequency can be set by the therapist
- Conveniently arranged display
- Over 25 clinical treatment suggestions
- 100 free programmable memory positions
- Special induction electrodes (superficial thermal burden low)
- Indication and monitoring of the power output to ensure optimum energy transmission
- Easy-to-adjust electrode arms that can be brought into position very easily.
- Opportunity to use two circuplodes simultaneously

13.5

MANUAL OPERATION

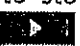
SCENARIO 2-SMART MODE IS OFF

8. Press  to accept and go back to the treatment screen.

In the navigation bar of the treatment screen, the  button will be available to store these parameters as a favorite. Press  to start treatment.

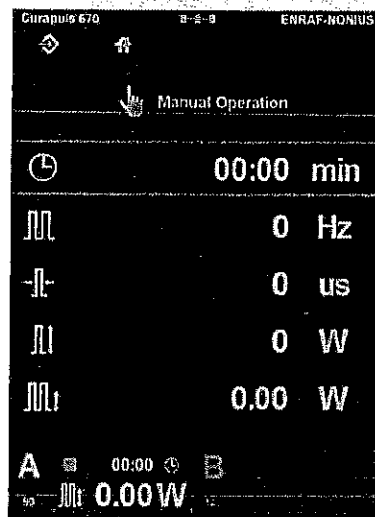The Channel tab will show


-   or  to indicate whether the treatment is running, paused or stopped.
- A countdown timer which indicates how much time is left until completion of the treatment.
- which electrode is connected (90mm, 140mm or E for Ellipse).
- the actual Mean power delivered to the patient.

13.4

Treatment stops automatically when the treatment time has elapsed or can be stopped by pressing the Stop key  to stop the active channel or press the  button to stop both active channels at the same time.

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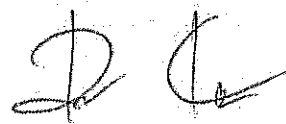13 SPECIFICATIONS 13**TECHNICAL DATA**

Mains voltage:	100 - 240 Volt
Frequency:	50/60 Hz
Max, Power consumption:	400 VA
Patient leakage current:	typically, 1 μ A
Idem single fault condition:	typically, 2 μ A

Dimension	(Length x Width x Height)	(Weight)
Main Unit	48 x 56 x 110 cm	Max. 27,2 kg
One Electrode Arm	-	3 kg
Electrode Cable	150 cm	-

Output Power		
Base Frequency:	27.12 MHz	13.2
Pulse duration:	65 - 400 μ s	} 13.3
Pulse frequency:	26 - 800 Hz	
Pulse power:	0 - 200 W	
Mean power:	0 - 64 W	
Treatment time:	0 - 30 minutes	13.4

Technical modifications reserved!

SAFETY AND PERFORMANCE STANDARDS

IEC 60601-1 General requirements for the safety of electrical medical systems.

Class I
Electrode classified as Type BF

IEC 60601-2-3 Particular requirements for the safety of shortwave therapy equipment.


IEC 60601-1-3 Non-ionizing radiation.

Medical device classification

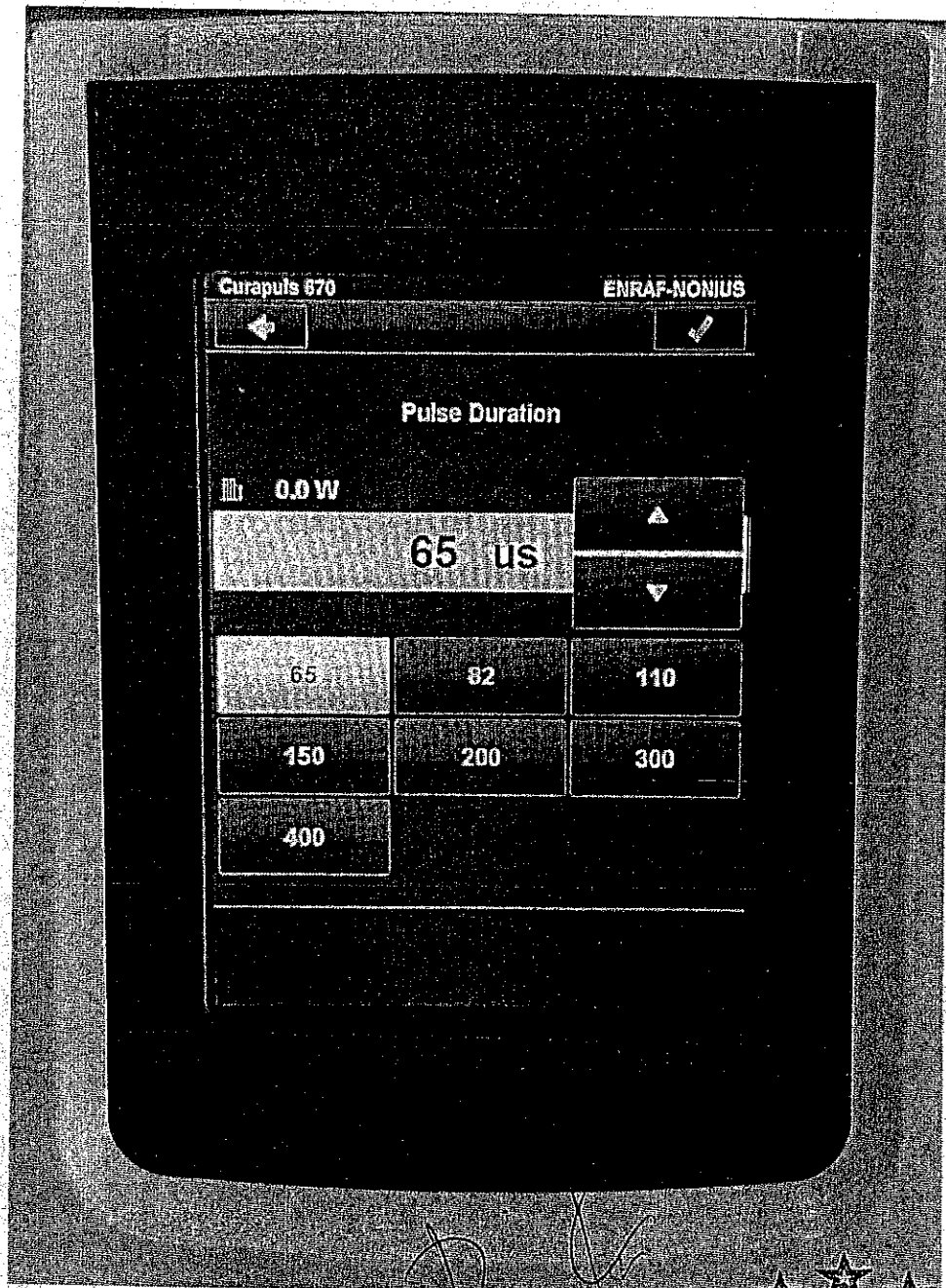IIa; Rule 9 Annex IX of 93/42/EEC
This equipment complies with all requirements of the Medical Device Directive (93/42/EEC).

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